




Zyno Medical LLC.  
177 Pine Street  
Natick, MA 01760  
USA

Phone: 1-508-650-2008  
Fax: 1-508-650-2006

## Z-800WF Infusion Pump

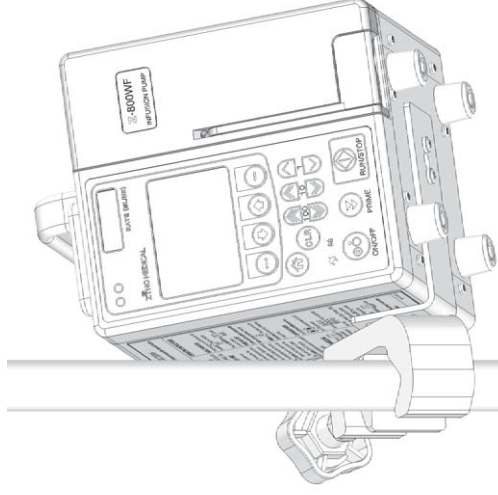

## INSTRUCTIONS FOR USE

Zyno Medical LLC  
177 Pine Street  
Natick, MA 01760  
1-508-650-2008  
Fax: 1-508-650-2006  
Web: [www.zynomed.com](http://www.zynomed.com)

**⚠ WARNING:**

Only use Zyno Medical provided IV sets with the Z-800WF Infusion Pump. There are risks associated with using any IV Sets other than Zyno Medical IV Sets with this device. Zyno's warranty for its device will be null and void and Zyno will assume no responsibility for any incidents that may occur if the device is not utilized strictly in accordance with its product labeling.

**SAFETY STANDARD**

The Z-800WF Infusion Pump meets all applicable safety standards for volumetric infusion pump devices, corresponding to IEC 60601-1 and IEC 60601-2-24.

Copyright © 2017 Zyno Medical, LLC  
Part number 800WF-IFU-4602

This page is intentionally left blank

Table of Contents

INTRODUCTION ..... 5

ABOUT THE DEVICE ..... 5

FEATURES ..... 6

SYMBOLS ..... 7

GETTING STARTED ..... 8

WARNINGS & CAUTIONS ..... 8

DEVICE DESCRIPTION ..... 11

INSTALLATION PROCEDURE ..... 14

PREPARING AN INFUSION ..... 15

LOADING PRIMARY ADMINISTRATION SET ..... 16

INFUSION MODE INFORMATION ..... 18

CONTINUOUS MODE ..... 18

TIME/VTBI PROGRAMMING ..... 26

INFUSION COMPLETE ..... 28

TPN MODE INFUSION ..... 29

10-STEP MODE INFUSION ..... 31

INTERMITTENT MODE INFUSION ..... 34

BLOOD INFUSION ..... 37

MANAGING INFUSION ..... 39

PRIME THE ADMINISTRATION SET USING PRIME KEY ..... 39

KEYPAD LOCKOUT ..... 40

CLEAR VOLUME INFUSED ..... 41

POWERING OFF ..... 42

CHANGING PRIMARY SOLUTION CONTAINER ..... 43

CHANGING AND RELOADING IV SET DURING INFUSION ..... 44

PROTOCOL MODE INFUSION ..... 45

SAVE INFUSION PARAMETERS AS PROTOCOL ..... 46

ZYNOFLO INFORMATION ..... 47

ZYNOFLO WIRELESS COMMUNICATION CONNECTION ..... 47

ZYNOFLO AND PATIENT QUERY COMMUNICATION ..... 49

OPTIONAL: MANUAL COMMUNICATION WITH ZYNOFLO ..... 51

ENABLING CAREGIVER ID FOR INFUSIONS ..... 53

This page is intentionally left blank

USING 10-STEP MODE WITH ZYNOFLO ..... 56

ALARMS AND TROUBLESHOOTING ..... 57

DEFINITIONS..... 57

ALARMS ..... 58

WARNINGS ..... 60

ERRORS ..... 62

MAINTENANCE..... 63

SPECIFICATIONS..... 63

CONFIGURABLE SETTINGS..... 65

STORAGE AND TRANSPORTATION ..... 82

BATTERY CARE AND MAINTENANCE ..... 83

CLEANING ..... 86

INSPECTION REQUIREMENTS ..... 87

SERVICE INFORMATION ..... 89

WARRANTY ..... 90

APPENDIX..... 91

ALARM TESTING PROCEDURE..... 91

APPROVED ADMINISTRATION SETS ..... 95

ACCESSORIES LIST ..... 97

**ACCESSORIES LIST**

<b>PART #</b>	<b>ITEM</b>	<b>Description</b>	<b>NOTE</b>
6601B	Pump Pole Clamp	45 Degree Pole Clamp	Standard Accessory
6602A	Power Cord	Removable 8 feet Hospital Grade AC power cord	Standard Accessory
6617A	Power Cord	Removable 12 feet Hospital Grade AC power cord	Optional Accessory
6611B	Drip Sensor	Drip Chamber-Based Flow Sensor	Optional Accessory
6612B	Secondary Red Alarm/Warning Light	Secondary Red Alarm/Warning Light Visual Indicator (magnetic)	Optional Accessory
6638A	Secondary Blue Alarm/Warning Light	Secondary Blue Alarm/Warning Light Visual Indicator (magnetic)	Optional Accessory

**APPROVED ADMINISTRATION SETS (Continued)**

<b>Primary Filter Sets</b>	
<b>PART #</b>	<b>ADMINISTRATION SET DESCRIPTION</b>
B2-70071-DF-120	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Pinch clamp, Roller Clamp, 1.2 Micron Adult I.V. filter, Slide clamp, Needleless Injection Site at distal side, Male Luer Lock Adaptor, Luer Lock Tip protector
B2-70071-DF	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Pinch clamp, Roller Clamp, 0.22 micron IVEX filter, Slide clamp, Needleless Injection Site at distal side, Male Luer Lock Adaptor, Luer Lock Tip protector
B2-70072-F	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Back-Check Valve, Two Needleless Injection Sites, one at the proximal side, one at the distal side, Pinch clamp, Roller Clamp, 0.22 micron IVEX filter, Slide clamp, Male Luer Lock Adaptor, Luer Lock Tip protector

**INTRODUCTION**

**ABOUT THE DEVICE**

The Z-800WF Infusion Pump is intended to provide accurate delivery of parenteral fluids, blood and blood products to a human patient when administered by a qualified health care professional.

**User Qualification**

The Z-800WF Infusion Pump is intended for use at the direction or under the supervision of licensed physicians or certified healthcare professionals who are trained in the use of the Z-800WF Infusion Pump and the administration of parenteral fluids and drugs, blood and blood products. This training should emphasize patient safety and prevention of error.

This document provides directions for use of the Z-800WF Infusion Pump. To ensure safe usage, please read the entire instruction manual before using the device.

The Z-800WF Infusion Pump must only be operated utilizing Zyno Medical's proprietary administration sets. The sets are designed for use with the Z-800WF Infusion Pump as well as for gravity-flow stand-alone use. For specific IV Administration Set instructions, refer to the Directions For Use provided with the set. For priming and loading instructions, refer to "Loading Primary Administration Set" and "Loading Secondary Administration Set" sections of this document.

Contraindication: None known.

# INTRODUCTION

## FEATURES

### Flow Rates

The Z-800WF Infusion Pump flow rate range is from 1 to 1200 ml/h.

### Free Flow Protection

The Z-800WF Infusion Pump has a built in free flow clamp to prevent inadvertent free flow when the set is loaded in the pump.

The Z-800WF Infusion Pump also has an IV Set based anti-free-flow module to prevent inadvertent free flow when the set is unloaded from the pump.

**⚠ WARNING:** Make sure to only use 800WF IV Sets with IV Set based anti-free flow clamp, and make sure to close the roller clamp before removing administration set from Z-800WF Infusion Pump.

### Occlusion Pressure

The Z-800WF Infusion Pump provides 14 levels of adjustable downstream occlusion alarm thresholds between 4 psi and 30 psi.

### Secondary Infusions

Secondary infusions may be delivered at a specified secondary delivery rate and secondary volume-to-be-infused (VTBI) independent of the primary infusion parameters. When secondary VTBI is greater than zero, the pump will always execute secondary infusion first. Automatic changeover occurs to the primary infusion parameters when the secondary infusion is complete. A Zyno Medical proprietary primary administration set with a back check valve must be used.

### System Configuration

The system configuration mode allows qualified personnel to customize device settings.

### Tamper Resist

The Tamper Resist feature provides a quick, one touch lockout of the front keypad.

### Volume-To-Be-Infused (VTBI)

The volume-to-be-infused (VTBI) range is from 1 to 9999 ml in 1 ml increments.

# APPENDIX

## APPROVED ADMINISTRATION SETS

### ⚠ WARNING:

Use only administration sets labeled as Zyno Medical with the Z-800WF Infusion Pump System. The use of any other set for use with Z-800WF system may cause improper instrument operation, resulting in inaccurate fluid delivery or other potential hazards.

The following lists some of the most commonly used administration sets approved for use with the Z-800WF Infusion Pump System. New administration sets configurations are added frequently. For complete administration set configurations, please visit Zyno Medical's web site at [www.zynomed.com](http://www.zynomed.com).


### ⚠ CAUTION:

It's recommended to replace Zyno Administration set within 72 hours.

Primary Needle-less Injection Port Sets	
PART #	ADMINISTRATION SET DESCRIPTION
B2-70071-P	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Back-Check Valve, Needleless Injection Site at proximal side, Pinch clamp, Roller clamp, Male Luer Lock Adaptor, Luer Lock Tip protector
B2-70071-D	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Pinch clamp, Roller Clamp, Slide clamp, Needleless Injection Site at distal side, Male Luer Lock Adaptor, Luer Lock Tip protector
B2-70072	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Back-Check Valve, Two Needleless Injection Sites, one at the proximal side, one at the distal side, Pinch Clamp, Roller Clamp, Slide clamp, Male Luer Lock Adaptor, Luer Lock Tip protector
B2-70070	105" (266cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Pinch clamp, Roller clamp, Male Luer Lock Adaptor, Luer Lock Tip protector
Secondary Sets	
PART #	ADMINISTRATION SET DESCRIPTION
A2-80075	40" (101cm) Long, Fluid path is sterile, non-Pyrogenic, non-DEHP, Latex-free; Spike tip protector, Universal spike, Drip chamber, Roller clamp, Male Luer Lock Adaptor, Luer Lock Tip protector, Bag Hanger

**ALARM TESTING PROCEDURE (Continued)**

**Battery Test (Continued)**

6. Verify the following:
  - Pump continues to operate during Low Battery warning.
  - The Battery indicator  turns red.
  - The warning audio tone and a visual message “Low Battery, Plug in AC power” displays every 10 seconds.
7. Record the time when the Battery Empty alarm displays. Record the VINP, Pri RATE and Pri VTBI parameters at the time of the alarm. Verify the following:
  - Battery Empty alarm continues to sound until user acknowledgement.
  - Infusion stops.

**BATTERY EMPTY**  
Press **STOP** to confirm

8. Allow the Battery Empty Alarm to continue to sound.
9. Record the time the pump shut itself off.
10. Connect the device to an approved AC power outlet.
11. Turn on the device. Verify 12-14 below.
12. All pump configuration parameters are preserved.
13. All current infusion parameters are preserved.
14. If any one of the following is true, contact a qualified service personal to replace the battery:
  - Time interval between the Low Battery Warning and Battery Empty Alarm is less than 15 minutes.
  - Time interval between the Battery Empty Alarm and the pump shut down is less than 2 minutes.
  - Pump configuration setting changed after battery depletion.
  - Any of the current infusion parameters is not preserved.

**SYMBOLS**



Electrical Shock Protection Rating: Type BF.

**IPX1**

Protection against fluid ingress: Drip Proof.



Attention: Refer to accompanying documentation.

**Rx Only**

Federal (U.S.A) law restricts this device to sale by or on the order of a physician.















Fuse: 2X2TA.

## GETTING STARTED

### GETTING STARTED

#### WARNINGS & CAUTIONS

The following Warnings and Cautions should be strictly followed to avoid harm to patients and pump operators:

-  **WARNING:** Zyno Medical assumes no responsibility for incidents that may occur if its product is not used in accordance with its product labeling.
-  **WARNING:** Only use Zyno Medical provided IV Sets with the Z-800WF Infusion Pump.
-  **WARNING:** The Z-800WF Infusion Pump operation is strictly limited to trained operators whose competency in safe Z-800WF Infusion Pump operation and safe IV therapy practices has been tested and proven.
-  **WARNING:** Make sure the pump is stable by fastening securely to an IV Pole, or resting on a flat surface. IV Poles vary in quality and stability. Avoid fastening the pump too high on the pole, and test for stability before using.
-  **WARNING:** Prior to use, always verify the proper function of the display, audible and visual alarms.
-  **WARNING:** Verify there are no kinks in the tubing when loading into the pump.
-  **WARNING:** Always read and follow the instructions that accompany the fluid container and IV Administration Sets you are using. Carefully follow the instructions in this document for loading, removing, and reloading the IV Set. Adjust the pumping section of the IV Set every 24 hours, and replace the IV Set within 72 hours.
-  **WARNING:** To prevent free flow, make sure to load the IV Set in the Infusion Pump before connecting the IV set to a patient.
-  **WARNING:** To prevent free flow, make sure to disconnect the IV Set from the patient before unloading the IV set from the Z-800WF Infusion Pump.
-  **WARNING:** Disconnect the IV Set from the patient before purging air bubbles out of IV Tubing.
-  **WARNING:** Make sure to close the roller clamp before removing IV Administration Set from the Z-800WF Infusion Pump.
-  **WARNING:** Do not over-program the VTBI. Program the actual amount of the fluid in the IV bag.

## APPENDIX


### ALARM TESTING PROCEDURE (Continued)

#### No-Clamp Alarm Test

1. Load an administration set with no air bubble into the pump. Make sure the IV set based clamp was NOT installed in pump. Close the pump door.
2. Program the primary flow RATE at 500 ml/h, and the primary VTBI at 100 ml.
3. Press the RUN/STOP key to start the infusion.
4. The infusion status indicator will turn blinking red, audio alarm will sound, and the Main Display will show “NO CLAMP”. The infusion will stop.
5. Press the RUN/STOP key again to acknowledge the alarm condition. The Z-800WF Pump will be in PAUSE state. The “PAUSE” message will appear on the display screen.
6. Install the IV set based clamp in pump. Press the RUN/STOP key again to resume infusion.



#### Battery Test

1. Connect the Z-800WF pump to an approved AC power outlet for at least 18 hours to allow the battery to fully charge.
2. Unplug the power cord from the AC power. Turn on the device. Verify that the battery indicator  is ON.
3. Set a primary infusion with the following:
  - Pri RATE = 125 ml/h
  - Pri VTBI = 1000ml/h
4. Start the infusion. Record the infusion starting time.
5. Record the time when the Low Battery Warning is presented.



ALARM TESTING PROCEDURE (Continued)

Occlusion Alarm Test

1. Program a primary infusion rate for 200 ml/h and VTBI for 50ml.
2. Press the **RUN/STOP** key to start the infusion.
3. Close the roller clamp on the administration set located directly at the distal side of the pump.
4. After a few seconds, the infusion status indicator will turn blinking red, audio alarm will sound, and the Main Display will show alarm screen with "Occlusion". The infusion will stop.
5. Press the **RUN/STOP** key to acknowledge the alarm condition. The Z-800WF Pump will be in PAUSE state. The "PAUSE" message will appear on the screen.
6. Open the roller clamp to release pressure. Press the **RUN/STOP** key again to resume infusion.

**Time Delay to Occlusion**

The maximum time for activation of the downstream occlusion alarm at the minimum flow rate of 1ml/hr is around 5 minutes at the minimum occlusion threshold setting. It is more than 1 hour at the maximum occlusion alarm threshold setting.

The maximum time for activation of the downstream occlusion alarm at the intermediate flow rate of 25ml/hr is around 20 seconds at the minimum occlusion threshold setting. It is around 3 minutes at the maximum occlusion alarm threshold setting.

**Occlusion Bolus Volume**

The maximum bolus volume generated as a result of operation at 25 ml/hr and reaching the minimum downstream occlusion alarm threshold is 0.25ml. The maximum bolus volume generated as a result of operation at 25 ml/hr and reaching the maximum downstream occlusion alarm threshold is 0.8mL.






**Note:** At slower infusion rates, the occlusion alarm may take longer to trigger.

WARNINGS & CAUTIONS (Continued)

- ⚠ **WARNING:** After starting an infusion, make sure drops are falling in the Drip Chamber. If no drops are falling, make sure the Roller Clamp is open. If the Roller Clamp is open and still no drops are falling, replace and dispose of the IV Administration Set.
- ⚠ **WARNING:** The Air In-Line Sensor cannot recognize the introduction of air at 3-way Stopcocks, Infusion Ports, and other Lines/Tubes below the Infusion Pump.
- ⚠ **WARNING:** Do not operate this device in the presence of Flammable Anesthetics mixture with Air, Oxygen or Nitrous Oxide.
- ⚠ **WARNING:** Do not expose the Z-800WF Infusion Pump to X-Rays, Gamma Rays or other Radiation, or to strong Electric or Magnetic Fields.
- ⚠ **WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.
- ⚠ **WARNING:** The manufacturer cannot assure the accuracy and/or safety of the Infusion Pump if it is not regularly maintained as recommended in the Z-800WF Infusion Pump Instructions For Use Manual.
- ⚠ **WARNING:** Flow rate accuracy can be affected by one or a combination of the following factors: fluid viscosity; fluid temperature; head height; back pressure; incompatible IV sets or compatible set with unusually small or large diameters or unusually stiff materials.
- ⚠ **CAUTION:** Please read the entire contents of this manual before using the Z-800WF Infusion Pump.
- ⚠ **CAUTION:** USA Federal and Canadian laws restrict this device to sale by or on the order of a physician.
- ⚠ **CAUTION:** There are no user serviceable parts inside. Refer all service, repair, and calibration to qualified technical personnel. Do not make unauthorized modifications.
- ⚠ **CAUTION:** To avoid mechanical or electronic damage, do not steam, autoclave or immerse the pump in any fluids or cleaning solutions, and do not spray such fluids directly on the pump. Always disconnect electrical power cord from outlet before cleaning to prevent electrical shock. The IPX1 rating means that the pump is protected against vertical dripping water.
- ⚠ **CAUTION:** Do not attempt to infuse two fluids simultaneously using the Z-800WF Infusion Pump.

## WARNINGS &amp; CAUTIONS (Continued)

-  **CAUTION:** Operating the Z-800WF Infusion Pump near equipment that radiates high-energy radio frequencies (electrosurgical/cauterizing equipment, portable radios, cellular telephones, etc.), or under direct sunlight towards top of infusion pump, may cause false alarm conditions. If this happens, reposition the device away from the source of interference or use an appropriate clinical alternative.
-  **CAUTION:** Always verify displayed infusion parameters (Primary Rate, Primary VTBI, Secondary Rate, Secondary VTBI) with the prescription before starting infusion.
-  **CAUTION:** Grounding reliability can only be achieved when equipment is connected to an equivalent receptacle marked "hospital only" or Hospital grade.

APPENDIX



## ALARM TESTING PROCEDURE

## Air-in-Line Alarm Test

1. Load an empty administration set into the pump. Make sure the IV set based clamp was installed correctly in pump. Close the pump door.
2. Program the primary flow RATE at 500 ml/h, and the primary VTBI at 100 ml.
3. Press the **RUN/STOP** key to start the infusion.
4. The infusion status indicator will turn blinking red, audio alarm will sound, and the Main Display will show "AIR-IN-LINE". The infusion will stop.
5. Press the **RUN/STOP** key again to acknowledge the alarm condition. The Z-800WF Pump will be in PAUSE state. The "PAUSE" message will appear on the display screen.
6. Clear the Air-in-Line condition. Press the **RUN/STOP** key again to resume infusion.

**Door Open Alarm Test**

1. Start a primary infusion. During operation, open the pump door.
2. The infusion status indicator will turn blinking red, the audio alarm will sound, and the Main Display will show "Door Open!". The infusion will stop.
3. Press **RUN/STOP** key to acknowledge the alarm condition. The Z-800WF Pump will be in PAUSE state. The "PAUSE" message will appear on the display screen.
4. Close pump door. Press the **RUN/STOP** key again to resume infusion.

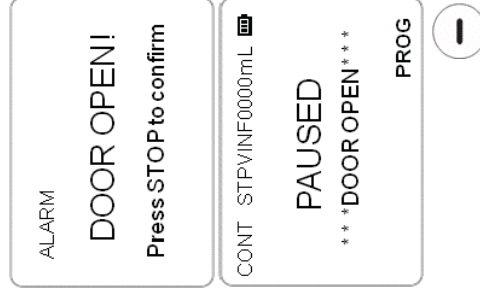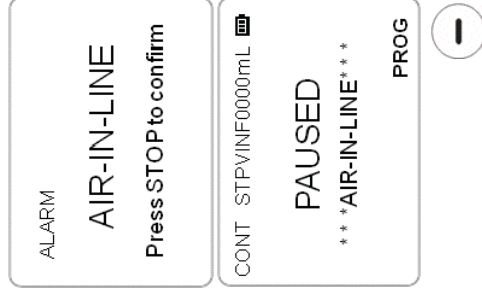

## MAINTENANCE

### WARRANTY

Zyno Medical LLC. (hereinafter referred to as “Zyno Medical”) warrants that:

- A. Each new Zyno Medical Z-800WF infusion pump is free from defects in material and workmanship under normal use and service for a period of one (1) year from the date of delivery by Zyno Medical to the original purchaser.
- B. Each new accessory (including batteries) is free from defects in material and workmanship under normal use and service for a period of ninety (90) days from the date of delivery by Zyno Medical to the original purchaser.

If any product requires service during the applicable warranty period, the purchaser should communicate directly with their relevant account representative to determine the appropriate repair facility. Except as provided otherwise in this warranty, repair or replacement will be carried out at Zyno Medical’s expense. The product requiring service should be returned promptly, properly packaged and postage prepaid by purchaser. Loss or damage in return shipment to the repair facility shall be at purchaser’s risk.

In no event shall Zyno Medical be liable for any incidental, indirect or consequential damages in connection with the purchase or use of any Zyno Medical product. This warranty shall apply solely to the original purchaser. This warranty shall not apply to any subsequent owner or holder of the product. Furthermore, this warranty shall not apply to, and Zyno Medical shall not be responsible for, any loss or damage arising in connection with the purchase or use of any Zyno Medical product which has been:

- (a) repaired by anyone other than an authorized Zyno Medical service representative;
- (b) altered in any way so as to affect, in Zyno Medical’s judgment, the product’s stability or reliability;
- (c) subjected to misuse or negligence or accident, or which has had the product’s serial or lot number altered, affected, or removed;
- (d) improperly maintained or used in any manner other than in accordance with the written instructions furnished by Zyno Medical.


This warranty is in lieu of all other warranties, express or implied, and of all other obligations or liabilities of Zyno Medical, and Zyno Medical does not give or grant, directly or indirectly, the authority to any representative or other person to assume on behalf of Zyno Medical any other liability in connection with the sale or use of Zyno Medical products.

Zyno Medical DISCLAIMS ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR OF FITNESS FOR A PARTICULAR PURCHASE OR APPLICATION.

## GETTING STARTED


### DEVICE DESCRIPTION

#### Pump Front View – Door Closed

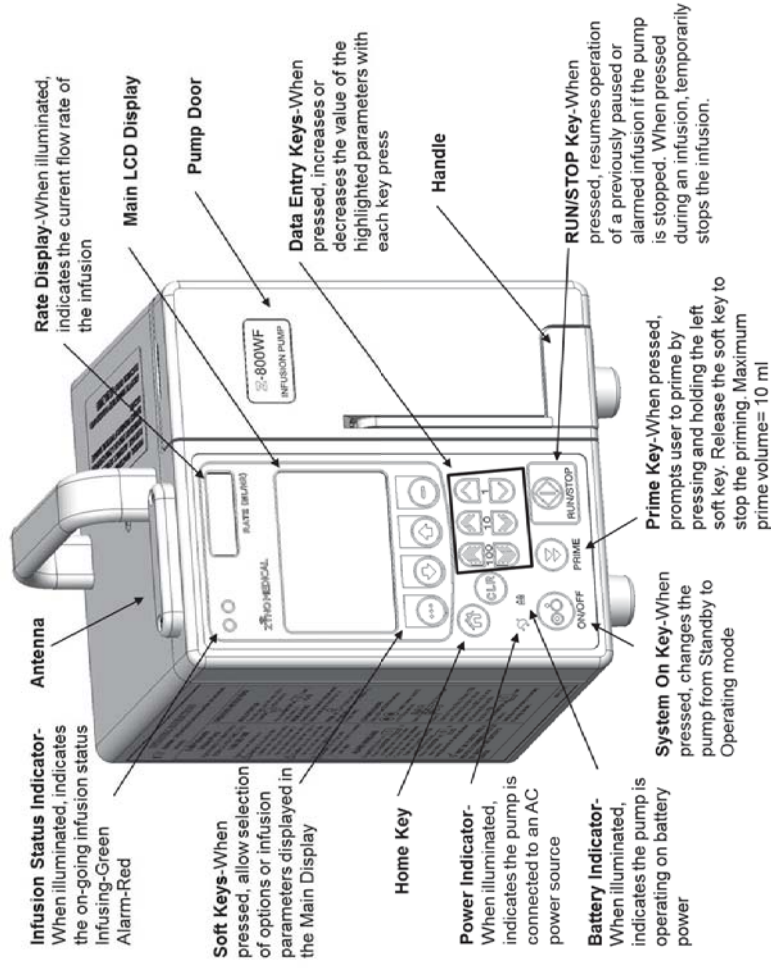

**DEVICE DESCRIPTION (Continued)**

**Pump Front View - Door Open**

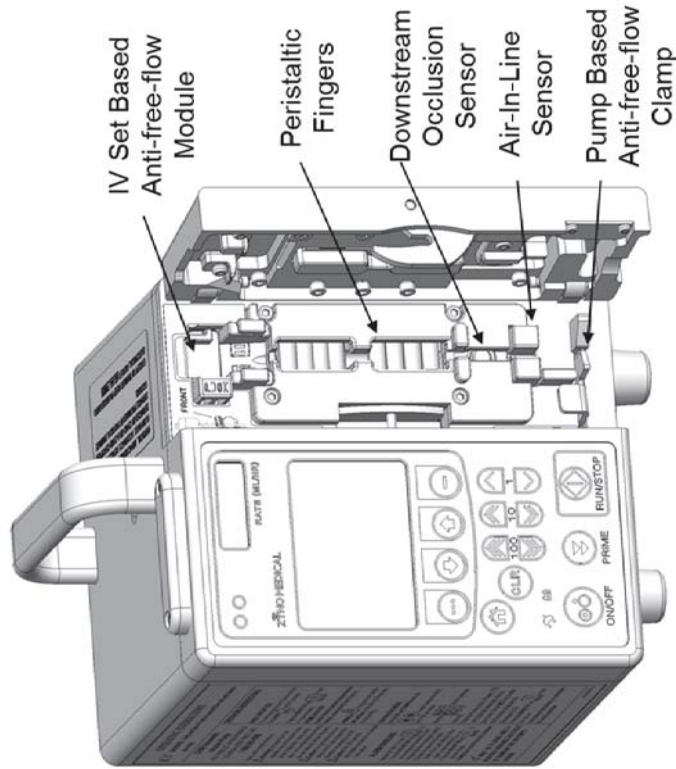

**SERVICE INFORMATION**

If a Z-800WF Pump fails to respond as described in this Instruction for Use and the cause cannot be determined, do not use the affected instrument. Contact qualified service personnel.

Within the United States, application and service information may be obtained by writing to Zyno Medical LLC. at:

Zyno Medical LLC.  
 177 Pine Street  
 Natick, MA 01760  
 ATTN: Instrument Service

Within the United States, information or assistance may be obtained by calling the Zyno distributor who provided the pump, or by calling Zyno Medical at (866)-395-1988.

Outside of United States, service information, applications, and manuals may be obtained by contacting your local Z-800WF Infusion Pump distributor.

When submitting any request for service, include:



- A description of the difficulty experienced.
- Z-800WF Pump serial number.
- Instrument settings and solution(s) used.
- Description, model and lot number(s) of the administration sets in use.
- Message displayed at the time of difficulty.

If it is necessary to return the instrument for service, obtain a return authorization number prior to shipment. Carefully package the instrument (preferably in the original packaging), reference the return authorization information, and return it to the appropriate service or distribution center. Zyno Medical does not assume any responsibility for loss of, or damage to, returned instruments while in transit.

Product complaints or adverse incidents should be reported to the Zyno Medical Quality Assurance Department at the above address. With each complaint, please include the pump serial number and a full description of the difficulty encountered, including all settings, types of fluids, times, and alarm messages. Return the administration set used if possible. Contact the Zyno Medical Customer Service Department for an RMA number prior to return.

## MAINTENANCE

### INSPECTION REQUIREMENTS (Continued)

The value of the odometer is displayed at the “Information” section of the configuration menu. The “Information” section has multiple pages. The user needs to press the up/down   arrow keys to navigate to different pages.

Loading the factory default will not change the value of the odometer. Only the Zyno qualified personnel can reset the odometer value.

**Periodic preventive maintenance of the hardware (PM’s) must only be performed by Zyno certified personnel, and are essential for assuring the accuracy and safety of the pump. PM’s MUST BE PERFORMED AT LEAST ONCE PER YEAR, OR WHEN THE SCREEN DISPLAYS THE DUE FOR SERVICE MESSAGE, WHICHEVER COMES FIRST.** A service agreement may be obtained from Zyno Medical for the performance of all required maintenance.

**⚠ WARNING: The manufacturer cannot assure the accuracy and/or safety of the pump if it is not regularly maintained as recommended in the Z-800WF Instructions for Use Manual.**

```

SN: XXXXXX      VXXXXX
IV. SET         ZYNO-XX
PRESSURE       XX PSI
AIR            X-XXmL
P XXX P4 XXX  P30 XXX
BATT XXXX X. XX% XX%
QUIT
    
```


```

ODOMETER
0001
QUIT
    
```

## GETTING STARTED

### DEVICE DESCRIPTION (Continued)

#### Pump Back View

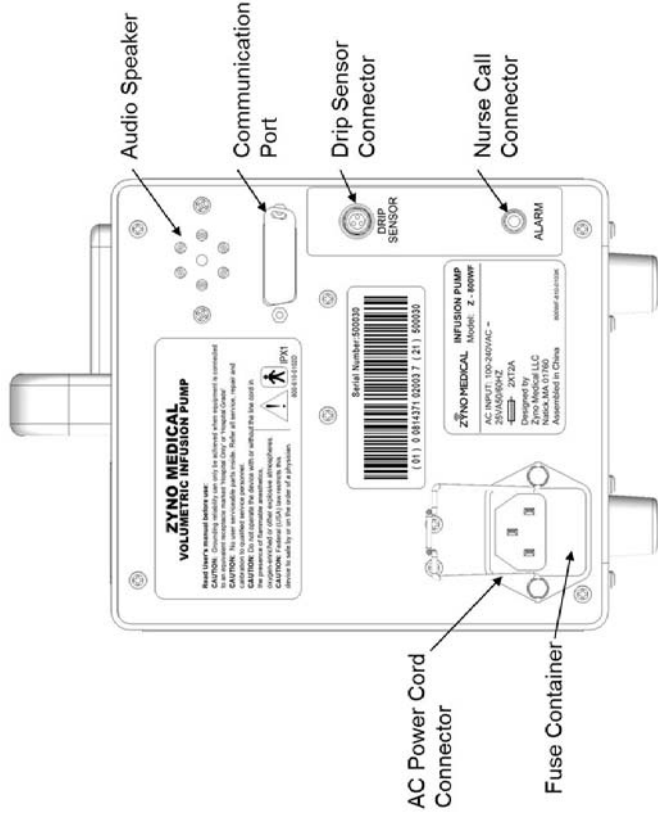

## GETTING STARTED

### INSTALLATION PROCEDURE

#### UNPACK THE PUMP

The Z-800WF Infusion Pump is supplied complete with

- A standard detachable, listed/certified IEC Hospital Grade electrical cord
- Pole Clamp pre-mounted at 45 degree angle to the pump
- Instruction for Use



1. Remove the pump module from its carton.
2. Inspect the pump.
3. Check to ensure the pump door operates freely.
4. Check for any loose parts.

**NOTE:** If any of the following conditions are observed, the Z-800WF Infusion Pump must be removed from use and inspected by qualified personnel:

- Look for any signs of physical damage from shipping.
- LED segments are not illuminated during system power-on self-test.
- Indicator lights do not illuminate.
- Audio tone does not sound.
- Main Display does not appear backlit, appears irregular, or has evidence of a row of pixels not functioning properly.

#### MOUNT PUMP TO IV POLE

Attach the pump to an IV Pole by turning the knob on the pole clamp clockwise, or place the pump on a flat, stable surface.

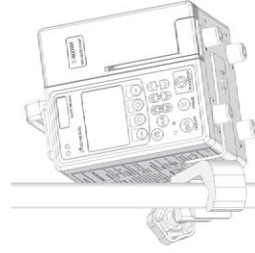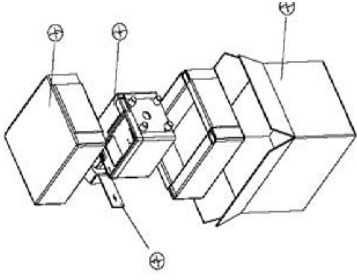

## MAINTENANCE

### INSPECTION REQUIREMENTS

To ensure the system remains in good operating condition, both regular and periodic inspections are required.

#### REGULAR INSPECTIONS:

Regular inspection consists of a visual inspection for damage and cleanliness, and performing the procedure described in the Start-Up Sequence section of this instruction for use before each usage of the instrument. Regular inspections are not covered under the contract or agreement offered by Zyno Medical and must be performed by the user.

#### REGULAR INSPECTIONS

PROCEDURE	FREQUENCY
<b>INSPECTION FOR DAMAGE:</b>	
Enclosure	Each usage
Power Cord	Each usage
<b>CLEANING</b>	
	As Required
<b>START-UP SEQUENCE</b>	
	Each usage

**NOTE:** If the instrument does not pass the regular inspection, the affected instrument must be removed from use and inspected by qualified service personnel.

#### PERIODIC PREVENTIVE MAINTENANCE:

The pump has a built-in maintenance cycle odometer. The odometer indicates how many liters of fluid can be accurately delivered before the pump must be serviced. This feature is designed to alert users for the required servicing timing for high-usage pumps.

When the value of the odometer approaches zero, the pump will display a warning message after power on, "Pump will be due for service. Please call 1-866-395-1988."

When the value of the odometer becomes zero or negative, the pump will display a warning message after power on, "Pump is due for service. Please call 1-866-395-1988."

**WARNING:** Failure to perform these inspections and maintenance may result in improper instrument operation.

Pump will be due for service soon. Please call 1-866-395-1988

QUIT

Pump is due for service. Please call 1-866-395-1988

QUIT

## CLEANING

**DO NOT** spray cleaning fluids directly onto the instrument or immerse the instrument in fluids.

**DO NOT** use solutions containing phosphoric acid (Foamy Q&A<sup>®</sup>), aromatic solvents (naphtha, paint thinner, etc.), chlorinated solvents\* (Trichloroethane, MEK, Toluene, etc.), ammonia, acetone, benzene, xylene or alcohol, other than as specified below.

**DO NOT** use hard or pointed objects to clean any part of the instrument.

Acceptable cleaning solutions are:

Warm water

Mild detergent (e.g., Manu-Klenz<sup>®</sup>)

\*\* 10% bleach solution (1 part bleach to 9 parts water)

\*\* Compublend<sup>™</sup> II

\*\* Envirocide<sup>®</sup>

\*\* 2% Glutaraldehyde in water

\*\* Hydrogen Peroxide 3%

\*\* 70% Isopropyl Alcohol

\*\* 2% Phenols in water (O-Syl 1:128, Pheno-Cen 1:256, Vesphene<sup>®</sup>)

\*\* 10% Povidone Iodine (betadine<sup>™</sup>)

\*\* Quaternaries 1:512

WEX-CIDE

1. Keep the instrument upright and do not allow any part of the instrument to become saturated with or submerged in fluid during the cleaning operation.
2. Use soft cloth dampened with warm water and a mild nonabrasive cleaning solution to clean all exposed surfaces. Do not spray any fluids directly on the instrument. For sanitizing or antibacterial treatment, use 10% bleach solution and water.



**WARNING:** Turn the instrument off and unplug the power cord from the AC power source before cleaning. Do not spray fluid directly onto the instrument. Do not steam autoclave, EtO sterilize, immerse the instrument or allow fluid to enter the instrument case. Failure to follow these instructions may result in an electrical hazard, damage to the instrument, and voided warranty coverage.




**CAUTION:** The solutions/solvents identified as NOT to be used can damage the surface of the instrument.

\*\* After application, rinse all surfaces with a water-dampened soft cloth.

## PREPARING AN INFUSION

### Powering On the System

1. Connect the Z-800WF Infusion Pump Power Cord to an AC power source.
2. Press and hold the **ON/OFF** key on the Z-800WF Infusion Pump for 2 seconds.
3. System self-test begins:
  - The diagnostics test causes all LED display segments and Status indicator lights to illuminate briefly.
  - The Power Indicator illuminates.
  - An Audio tone sounds.
4. The Main Display shows the Zyno Medical Model Type, Pump Serial Number, Software Version and pump safety configurations during the system power-on self-test.
5. Upon completion of system power-on self-test, a New Infusion prompt screen will be displayed prompting the user to choose either resume a previously interrupted infusion or start with a new infusion.
6. If the user elects to resume the previously interrupted infusion, the pump will enter into the Programming screen of the previous selected infusion mode and with the previous infusion parameters populated.
7. If the user elects to start a new infusion, the pump will prompt the user to main screen (see right image). For manual programming, the user should select **PROGRAM**. After selection, the pump will prompt user to select infusion mode. Upon selection, the pump will enter into the Infusion Programming screen of the selected Infusion Mode with all infusion parameters reset to zero. The user may choose one of the following infusion modes:
  - Continuous Mode with Rate/VTBI
  - Continuous Mode with Time/VTBI
  - TPN Auto Ramp Mode
  - 10 Step Mode Rate/VTBI
  - 10 Step Mode Time/VTBI
  - Intermittent
  - Blood Infusion

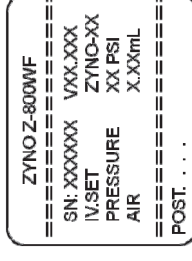

## LOADING PRIMARY ADMINISTRATION SET

### Preparing the Primary Solution Container

Prepare the primary solution container in accordance with the manufacturer's directions for use.






### Preparing the Primary Administration Set

Use only Zyno IV Administration Sets (refer to **Approved Administration Sets** section for a list of compatible sets). Open the administration set package, remove the set and close the roller clamp.

**⚠ WARNING: Make sure to close Roller Clamp before removing IV Administration Set from Z-800WF Infusion Pump.**

### Loading Primary Administration Set

1. Make sure the Anti-Free Flow Pinch Clamp is attached on the primary set and it's in open position. (see picture "Step 1")
2. Insert the Set Spike into prepared fluid container following accepted clinical procedure, and hang the container approximately 20 inches above the Infusion Pump.
3. Fill the Drip Chamber to 1/3 full by squeezing it.
4. Open the Roller Clamp slowly to prime tubing. Invert and tap the back Check Valve and any luer lock sites to clear the air from the IV Administration Set.
5. Close the Roller Clamp and the Pinch Clamp.
6. Close the pump door. Turn on the pump. After the pump has finished the power-on-self-test, open the pump door.
7. Using a thumb, firmly push the metal button of the Anti-Free-Flow Module, slide the Clamp into the Clamp Holder from the top (pay attention to the orientation of the clamp) and release the button. (See picture "Step 7")
8. Using a fingertip, firmly push the tubing into the gap of the tubing guides on the Pumping Fingers. (See picture "Step 8")
9. Using a fingertip, firmly push the tubing into the gap of the Air-in-Line Sensor. (See picture "Steps 9 & 10")
10. Align the tubing on top of the gap of the Free Flow Clamp. Do not force the tubing into the clamp. The tubing will be loaded into the Free Flow Clamp automatically when the pump door is closed.
11. Close the pump door by pushing down the pump door handle (See picture "Step 11")
12. Open the IV set roller clamp and confirm that there is NO FLOW in the IV set drip chamber. Connect the administration set to the patient's injection site.

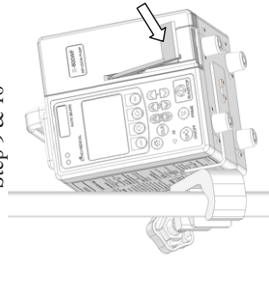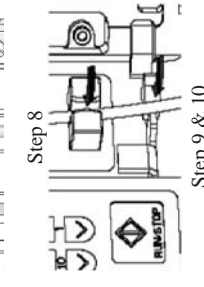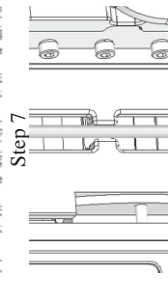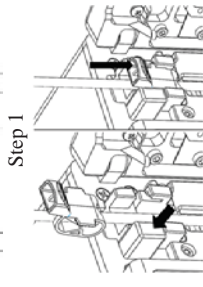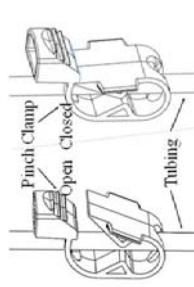

## BATTERY CARE AND MAINTENANCE (Continued)

### Battery Care

The battery capacity should be checked at least once every 12 months.

If the Z-800WF pump is to be stored at temperatures in excess of 86 °F (30 °C) for one or more months, the battery should be removed and placed in an environment between 50 – 86 °F (10-30 °C).

If the battery is to be stored for more than one year, it should be charged at least once per year to prevent leakage and deterioration in performance due to self-discharge.

When the battery is first being put into use, or has been out of use for one or more months, it will not have full capacity due to deactivation of reactants. Restore the battery to original performance by repeating one or two cycles of fully charging and fully discharging.

Some temporary reduction in capacity might become apparent if the battery is repeatedly discharged less than completely. One or two cycles of full discharge and full charge can restore full performance.

The Z-800WF Pump is shipped with a battery in a discharged condition. Connect the power cord to an AC receptacle and allow the battery to charge for 18 hours.

Whenever possible, leave the power cord connected to an external AC power source while operating.

**⚠ CAUTION:** Battery replacement should be performed by qualified service personnel while the instrument is not in use.

**⚠ CAUTION:** All pump configuration settings need to be verified and reset as needed after a complete discharge of battery.

**⚠ CAUTION:** DO NOT open, incinerate or short circuit battery. Worn out batteries must be disposed properly, according to local regulations.



**BATTERY CARE AND MAINTENANCE (Continued)**

**Battery Operating Time**

Battery run time is a function of the activity of the device. In the standard battery configuration of an 4500mAh battery, a fully charged new battery will provide approximately 8 hours of operation infusing at 125ml/h. As flow rate increases, the power consumption increases, the battery operating time will decrease accordingly. See *TABLE: Battery Operating Time* on the right. In the standard configuration, a fully discharged battery will return to a fully charged capacity in 5-6 hours in an ambient temperature between 50 – 86 °F (10 – 30 °C).

Table: Battery Operating Time

Flow Rate	Run Time
1ml/hr	10hrs
25ml/hr	10hrs
125ml/hr	8hrs
1200ml/hr	6hrs

**Battery Cycle Life and Aging**

As batteries get older and go through many charge/discharge cycles, batteries “wear out,” meaning that the chemicals and materials used to construct the cell break down. It is estimated that there will be a 30% capacity decrease of battery capacity over a 200 full discharge/charge cycles within 2 years of normal use. As the battery ages, the battery operating time will decrease.


**Partial Discharge/Recharge**

When a battery is partially discharged, then charged for less than the full time, differences between individual cell capacities result in cells completing charge at different times. If the full charge sequence is not then completed, the cell “mismatch” becomes progressively greater. This will be observed by user as low apparent run times and premature low battery warning and alarms. The lowered capacity is not permanent, but may require 2-3 full discharge/charge cycles to recover.

**⚠ CAUTION:** The battery capacity indicator display represents the total capacity of the battery measured by voltage. It only represents the percentage of battery capacity against its total capacity. As battery ages, the total capacity of the battery will decrease.

**LOADING PRIMARY ADMINISTRATION SET (Cont.)**

**LOADING IV SET INTO Z-800WF PUMP**

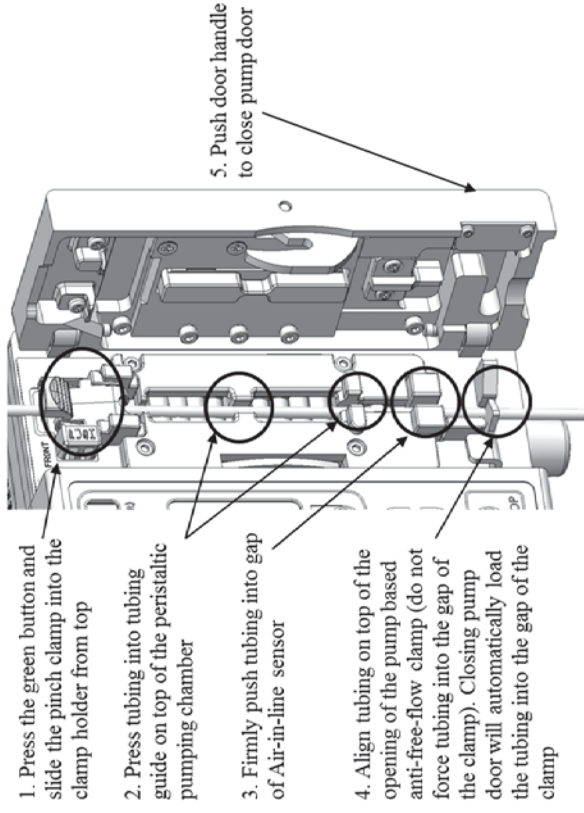

**⚠ WARNING:** Make sure to close roller clamp before removing administration set from Z-800WF pump.

## CONTINUOUS MODE

### CONTINUOUS MODE

#### CONTINUOUS MODE-PRIMARY INFUSION

##### Primary Infusion Programming Screen

The user may choose to program the continuous infusion either in RATE/VTBI or TIME/VTBI. The primary infusion programming screen contains the following display areas:

1. **Status Bar:** The Status Bar displays current operating mode, current pump state and volume infused.

- **CONT:** Indicates the current application mode is Continuous Mode Operation

- **VINF:** Indicates the Volume Infused during the current infusion

2. **Infusion Parameters:** -- Primary Flow Rate and Primary Volume to Be Infused

- **Pri RATE:** Primary Flow Rate

- **Pri VTBI:** Primary VTBI

3. **Keys:** Allows user to access pump configuration and secondary infusion programming screen

- **CONFIG:** Access configurable settings

- **SEC:** Access the secondary infusion programming

- **Up/Down Arrows:** Select parameter items

- **Home Key:** Access Infusion Mode

△ **WARNING:** Do not over-program VTBI. Program the actual amount of IV bag fluid.

##### Change Primary Flow Rate

Use Up/Down Arrow keys to highlight Pri RATE. Use the Data Entry keys to modify the corresponding digits of Pri RATE value.

**Example:** Programming Pri RATE = 125 mL/h

1. Use Up/Down Arrow keys to highlight Pri RATE.
2. Press the 100 Up data entry key **100** once to increase the 100 digit of Pri RATE to 100.
3. Press the 10 Up data entry key **10** twice to increase the 10 digit of Pri RATE to 20.
4. Press the single digit Up data entry key **5** five times to increase the single digit of Pri RATE to 5.

## BATTERY CARE AND MAINTENANCE

### Battery Type and Charging

The Z-800WF can operate on internal rechargeable battery power, enabling continued infusion when the patient is being transferred or during electrical power failure.

When the pump runs on battery power, the POWER indicator  is off, and the BATTERY indicator  is on.

The Z-800WF Pump is equipped with a standard configuration of 9.6 volt 4500mAh Nickel-Metal-Hydride battery. The battery charges whenever the pump is plugged into an AC outlet. The life expectancy of the battery is dependent on the amount of use, the depth of discharge, and the state of the charge that is maintained. Generally, the battery will have the longest life if the pump is always plugged in and the battery use is infrequent. Frequent use of battery power and insufficient battery charge cycles will significantly decrease the life of the battery. It is recommended that the battery be replaced after three years.

The quality of the battery is also a significant factor in determining battery life and runtime. The battery cannot be repaired and should not be opened. The battery may only be replaced with an approved battery from Zyno Medical. Use of any other brand may yield poor performance, and will invalidate the warranty.


The battery should be charged in a room with a temperature between 50 and 86 °F (10 – 30 °C) to minimize charge time and maximize battery life.

## MAINTENANCE

### STORAGE AND TRANSPORTATION

Store the pump away from excessive heat, cold, or humidity.

Keep the pump plugged into an AC outlet during storage, to ensure a fully charged battery when needed.

 The AC indicator light pictured left will be on whenever the Z-800WF Infusion Pump is plugged in.

When transporting the pump, please apply sufficient protection to the pump to prevent physical damages. It is recommended to use a box size of 10x10x12 for an individual pump. The upright orientation with lifting handle facing up is recommended during transportation.

## INFUSION MODE INFORMATION

### CONTINUOUS MODE-PRIMARY INFUSION (Continued)


#### Change Primary VTBI

Use the Up/Down Arrow keys to highlight Pri VTBI. Use the Data Entry keys to modify the corresponding digits of Pri VTBI value.

**Example:** Programming Pri VTBI = 250 ml  
Use the Up/Down Arrow keys to highlight Pri VTBI.

 Press the 100 Up data entry key **100** twice to increase the 100 digit of Pri VTBI to 200.

 Press the 10 Up data entry key **10** five times to increase the 10 digit of Pri VTBI to 50.

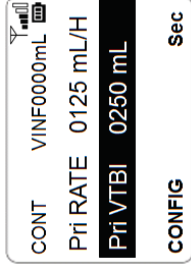

CONT	VINF0000mL	
Pri RATE	0125 mL/H	
Pri VTBI	0250 mL	
CONFIG		Sec

#### Start Primary Infusion

Verify the displayed infusion parameter entries (Primary Rate, Primary VTBI). If a clamp is engaged, remove the clamp. Press **RUN/STOP** key to start the infusion.



#### Stopping and Resuming a Primary Infusion

Press the **RUN/STOP** key during infusion. The infusion will be paused. The **PAUSE** screen will be displayed.

From the **PAUSE** state, press the **RUN/STOP** key to resume infusion.

#### Titrate Flow Rate or VTBI During a Primary Infusion

From the **PAUSE** screen, press the **PROG** key; the pump will display primary infusion programming screen. The user is then able to modify the primary RATE and primary VTBI parameters.



## CONTINUOUS MODE-PRIMARY INFUSION (Continued)

### Primary Infusion Execution

- During an infusion, the two Infusion Status Indicator LEDs will illuminate in green and toggle once per second.
- The primary infusion execution display contains the following contents:  
**Status Bar:** The Status Bar displays the current operating mode, current pump state, and volume infused.
  - **PRI:** Primary infusion
  - **RUN:** Pump RUN state
  - **TL:** Time Left in HH:MM format for the current infusion


### Infusion Parameters: Current infusion parameters.

- **RATE** field displays primary flow rate.
- **VTBI** field counts down to show remaining VTBI.
- **VINF** field counts up to show volume infused.
- 3 minutes before infusion completes, a short audio prompt tone will sound twice. Along with the audio prompt, the screen will display "INFUSION NEAR END" message. This alert will repeat every 5 seconds until the infusion is complete.
- At completion of the infusion, an audio prompt sounds and the screen will display "INFUSION COMPLETE – KVO." This alert will repeat every 5 seconds until user intervention. During KVO state, the Flow Rate Indicator will change to display KVO flow rate.

### Clearing the Volume Infused During a Primary Infusion:

Press the RUN/STOP key to PAUSE the primary infusion. Exit the PAUSE screen by pressing the PROG key. From the **Primary SETUP** screen, press and hold the CLR key. An audio/visual prompt will be presented requesting user confirmation to clear VINF. Press the YES key to confirm. The VINF field will be reset to 0 ml. The user can press the NO key to abort the action.

**NOTE:** The VINF field will record cumulative infusion volume infused unless cleared by the user pressing the CLR key or turning the pump off/on and starting a new infusion.

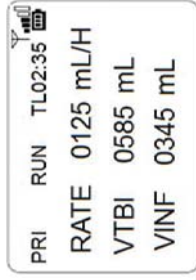

## CONFIGURABLE SETTINGS (Continued)



### Load Factory Default

The user can load the factory default settings in the configuration menu.

From the configuration page, pressing CLR key and **100** key at the same time will bring out the customization authorization screen.

Enter the password "1111" by pressing **1** and **10**

once and pressing **100** 11 times. Then press "SELECT". The pump will enter the customization page. Use the up/down **10** arrow keys to move to the LOAD DEFAULT option. Use **1** or **10** to remove the "\*" sign and choose "SELECT" to save the changes and return to the configuration menu. Use the up/down **10** arrow keys to move to the LOAD DEFAULT option and press "SELECT". A confirmation page will be displayed: "Confirm Restoration Of Factory Default Parameters?" Pressing "YES" would reset the pump to factory default parameters. Press "NO" to cancel the action.

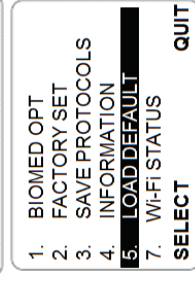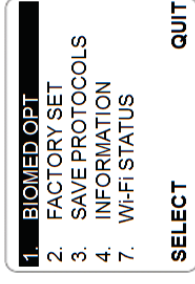






## CONFIGURABLE SETTINGS (Continued)




### Customize the FACTORY SET Menu

The user can choose to display or hide certain items in the "FACTORY SET" menu.




In the configuration menu, select "FACTORY SET."

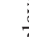In the "FACTORY SET" page, pressing the CLR key and  keys at the same time will bring out the customization authorization screen.

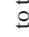Enter the password "1111" by pressing  and  once and pressing  11 times. Then press "OK." The pump will enter the customization page. The "\*" sign before an item means that the item will not be displayed in the "FACTORY SET" page. Use  and


 to toggle the "\*" sign before each item. Use the up/down   arrow keys to move to the selected item.


After the user has finished the customization, he or she can choose "SELECT" to save the changes and return to the "FACTORY SET" main page. Pressing "QUIT" will return to the "FACTORY SET" main page without saving the changes.

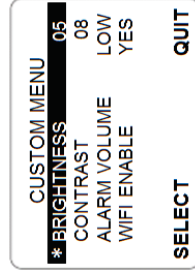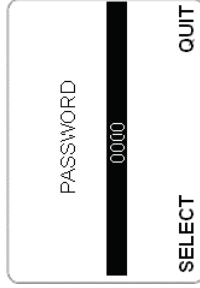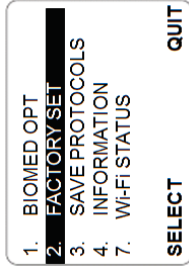


## LOADING SECONDARY ADMINISTRATION SET

1. Program and start the primary infusion using a check-valve primary administration set, as previously described.
2. Open a secondary administration set package, remove the set and close the set roller clamp.
3. Insert set spike into a prepared fluid container and hang secondary container, following accepted clinical procedures.
4. Fill the drip chamber to 1/3 full.
5. Open a secondary set and prime the set. Close the set roller clamp.
6. Attach a secondary set to the upper injection site on the primary set.
7. Ensure that no air bubbles are in the line.
8. Hang the secondary fluid container at least 8 inches above the primary solution container.

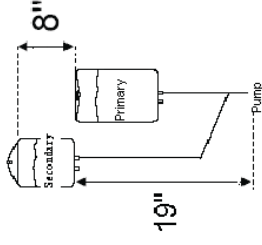

 **WARNING: Make sure to close roller clamp before removing administration set from Z-800WF pump.**

 **WARNING: The bottom of the secondary solution should be at least 19" above the top of the pump.**

 **WARNING: A minimum height differential between primary and secondary solutions of eight inches is essential for the safe operation of a primary/secondary infusion.**

## CONTINUOUS MODE-SECONDARY INFUSION

### Access Secondary Infusion Programming Screen



From the Continuous Mode Primary Infusion Programming screen, Press the Sec key to access the secondary infusion programming screen.

**NOTE:** The **SECONDARY INFUSION** option can be hidden if it is not required. To configure the Secondary Infusion options go to the **BIOMED OPT** Menu.

### Secondary Infusion Programming Screen

Similar to the primary infusion programming screen, the secondary infusion programming screen contains the following display areas:

1. **Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - **CONT:** Continuous Mode Operation
  - **VINF:** Volume Infused for the current infusion
2. **Infusion Parameters:** Secondary Flow Rate and Secondary Volume to Be Infused.
  - **Sec RATE:** Secondary Flow Rate
  - **Sec VTBI:** Secondary VTBI
3. **Keys:** Allow user to access pump configuration and primary infusion programming screen.
  - **CONFIG:** Access pump configuration
  - **PRI:** Access primary infusion programming
  - **Up/Down Arrows:** Select parameter items

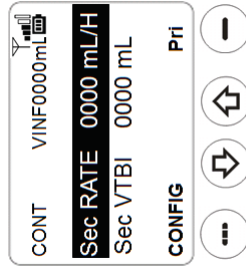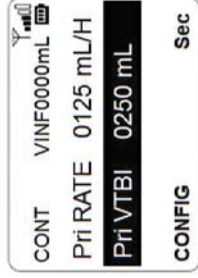



## CONFIGURABLE SETTINGS (Continued)

### Customize the BIOMED OPT Menu

The user can choose to display or hide certain items in BIOMED OPT page.


In the configuration menu, select "BIOMED OPT."

In the "BIOMED OPT" page, pressing the CLR key and keys at the same time will prompt the customization authorization screen.

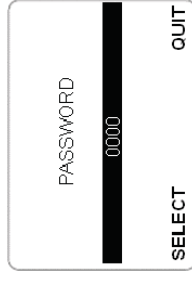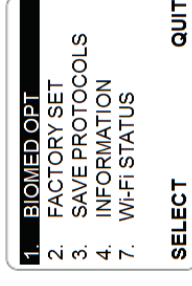

Enter the password "1111" by pressing and once and pressing 11 times. Then press "OK." The pump will enter the customization page. The "##" sign before an item means that the item will not be displayed in the "BIOMED OPT" page. Use and to toggle the "##" sign before each item. Use the up/down arrow keys to move to the selected item.

After the user has finished the customization, he or she can choose "SELECT" to save the changes and return to the "BIOMED OPT" main page. Pressing "QUIT" will return to the "BIOMED OPT" main page without saving the changes.

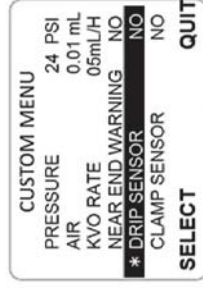

# MAINTENANCE

## CONFIGURABLE SETTINGS (Continued)

### Customize the Load Protocol Menu

The user can customize the protocol menu to display or hide certain protocols .


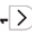
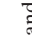
In home page, select **"PROTOCOL"**.

In the **"LOAD PROTOCOLS"** page, pressing the




CLR key and  at the same time will prompt the customization authorization screen.

Enter the password "1111" by pressing  and **10**

once and pressing **100** 11 times. Then press "OK." The pump will enter the customization page. The "\*" sign before an item means that the item will not be

displayed in the "Protocol" page. Use  and  to toggle the "\*" sign before each item. Use the up/down  arrow keys to move to the selected item.

After the user has finished the customization, he or she can choose "SELECT" to save the changes and return to the **"LOAD PROTOCOLS"** main page. Pressing "QUIT" will return to the **"LOAD PROTOCOLS"** main page without saving the changes.

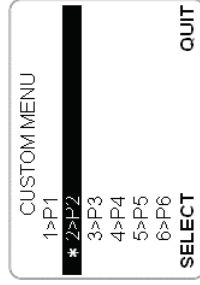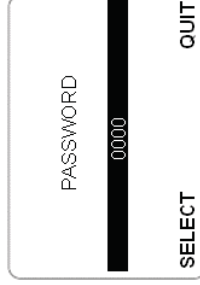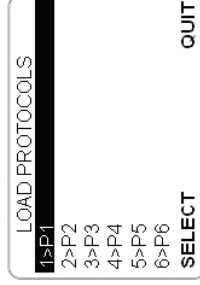

# INFUSION MODE INFORMATION


## CONTINUOUS MODE-SECONDARY INFUSION (Continued)

### Change Secondary Flow Rate

Use Up/Down Arrow keys to highlight Sec RATE. Use Data Entry keys to modify the corresponding digits of Sec RATE value.

**Example:** Programming Sec RATE = 275 mL/h

1. Use the Up/Down Arrow keys to highlight Sec RATE.
2. Press the 100 Up data entry key **100** twice to increase the 100 digit of Sec RATE to 200.
3. Press the 10 Up data entry key **10** seven times to increase the 10 digit of Sec RATE to 70.
4. Press the single digit Up data entry key **5** five times to increase the single digit of Sec RATE to 5.

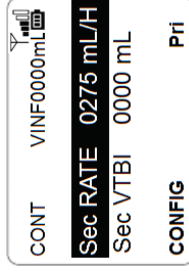


### Change Secondary VTBI

Use the Up/Down Arrow keys to highlight Sec VTBI. Use the Data Entry keys to modify the corresponding digits of Sec VTBI value.

**Example:** Programming Sec VTBI = 150 mL


1. Use the Up/Down Arrow keys to highlight Sec VTBI.
2. Press the **100 Up data entry key 100** once to increase the 100 digit of Sec VTBI to 100.
3. Press **10 Up data entry key 10** five times to increase the 10 digit of Sec VTBI to 50.

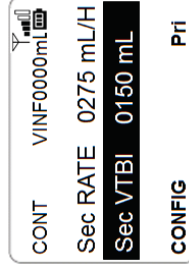

## CONTINUOUS MODE-SECONDARY INFUSION (Continued)

### Start Secondary Infusion

Verify displayed infusion parameter entries (Primary Rate, Primary VTBI, Secondary Rate, and Secondary VTBI). If a clamp is engaged, remove the clamp. Press **RUN/STOP** key to start the infusion.

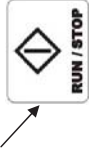

**NOTE:** Secondary infusion will be executed before primary infusion.

### Secondary Infusion Execution

- The two Infusion Status Indicator LEDs will illuminate in green and toggle once per second.
- The secondary infusion execution display contains the following contents:

Status Bar: The Status Bar displays current operating mode; current pump state; and volume infused.


- **SEC:** Secondary infusion
- **RUN:** Pump RUN state
- **TL:** Time Left in hours: minutes (HH:MM) format for the current infusion.

Infusion Parameters: Current infusion parameters

- **RATE** field displays secondary flow rate.
- **VTBI** field counts down to show remaining VTBI.
- **VINF** field counts up to show volume infused.

- Upon completion of secondary infusion, a switchover audio alert sounds 4 short beeps. The Main Display will switch-over to **PRI (primary) State**.

**⚠ WARNING: Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.**

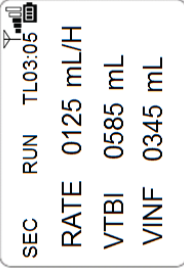

## CONFIGURABLE SETTINGS (Continued)

### Customize the Infusion Mode Menu

The user can customize the infusion mode menu to display or hide certain items.


Press the home key to access the infusion mode menu.

In the mode selection page, pressing CLR key and key at the same time will bring out the customization authorization screen.




Enter the password "1111" by pressing and once and pressing 11 times. Then press "SELECT." The pump will enter the customization page. The "\*" sign before an item means that the item will not be displayed in the mode selection settings.

Use and to toggle the "\*" sign before each item. Use the up/down arrow keys to move to the selected item.

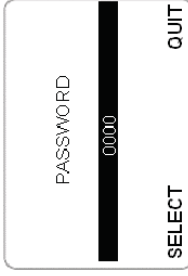

After the user has finished the customization, he or she can choose "SELECT" to save the changes and return to the infusion mode menu. Pressing "QUIT" will return to the infusion mode menu without saving the changes.

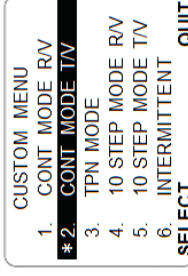







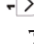
## MAINTENANCE

### CONFIGURABLE SETTINGS (Continued)

#### Customize the Configuration Menu



The user can customize the configuration menu to display or hide certain items.

From the configuration page, pressing CLR key and  key at the same time will bring out the customization authorization screen.

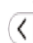Enter the password "1111" by pressing  and  and **10** once and pressing **100** 11 times. Then press "SELECT". The pump will enter the customization page. The "\*" sign before an item means that the item will not be displayed in the configuration settings. Use  and  to toggle the "\*" sign before each item.

Use the up/down   arrow keys to move to the selected item.

After the user has finished the customization, he or she can choose "SELECT" to save the changes and return to the configuration menu. Pressing "QUIT" will return to the configuration menu without saving the changes.

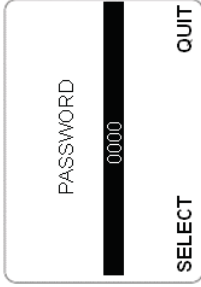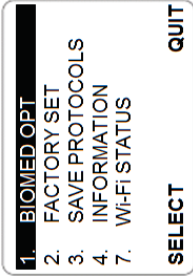

## INFUSION MODE INFORMATION

### CONTINUOUS MODE-SECONDARY INFUSION (Continued)


#### Stopping and Resuming a Secondary Infusion

Press the **RUN/STOP** key during infusion. The infusion will be paused. The **PAUSE** screen will be displayed.

From **PAUSE** state, press the **RUN/STOP** key to resume infusion.

#### Changing Rate or VTBI During a Secondary Infusion

From the **PAUSE** screen, press **PROG** key; the Z-800WF Infusion Pump will enter secondary infusion programming screen. The user is able to modify the secondary **RATE** and secondary **VTBI** parameters.

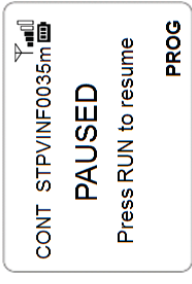

## TIME/VTBI PROGRAMMING

### Access TIME/VTBI Programming Option

The Z-800WF Infusion Pump provides TIME/VOLUME programming options for Continuous Mode Infusion. The user may choose the infusion programming parameter option in the pump Infusion Mode selection screen.

1. Press Home key to access the main page. If a confirmation page showing "Current infusion parameters will be lost. Want to continue?" is displayed, select YES to acknowledge.  
**Note:** Any unsaved parameters will be lost upon confirmation.

2. Select PROGRAM to access Infusion Mode menu.

3. In pump Infusion Mode menu, use the Up/Down Arrow keys to highlight **2. CONT MODE T/V** and press the **SELECT** key.

4. The display will switch to continuous mode primary infusion programming screen with TIME and VTBI parameters.

Current infusion parameters will be lost  
 Want to continue?

YES NO

PROGRAM  
 PROTOCOL  
 PATIENT QUERY  
**SELECT**






1. CONT MODE RV  
**2. CONT MODE T/V**  
 3. TPN MODE  
 4. 10 STEP MODE RV  
 5. 10 STEP MODE T/V  
 6. INTERMITTENT  
**SELECT**

## CONFIGURABLE SETTINGS (Continued)

### Enable/Disable Wireless connection

The Z-800WF pump offers an option to disable/enable wireless connection.

**Note:** the change will become effective after turning off the pump and turning on again.

1. From the configuration menu, Select **2. FACTORY SET**
2. In the **FACTORY SET** menu, highlight the **WIFI ENABLE** option by pressing the **Up/Down Arrow**.
3. Press the  or  key to toggle between YES and NO.
4. Press the **SAVE**  key to confirm the set value.  
 Press the **CANCEL** key  to back out. Press the **QUIT** key  again to exit the **CONFIGURATION** menu.

1. BIOMED OPT  
**2. FACTORY SET**  
 3. SAVE PROTOCOLS  
 4. INFORMATION  
 7. WI-FI STATUS  
**SELECT** QUIT

FACTORY SET  
 BRIGHTNESS 05  
 CONTRAST 08  
 ALARM VOLUME LOW  
**WIFIENABLE YES**  
**SAVE** CANCEL



## CONFIGURABLE SETTINGS (Continued)

### Adjusting the Alarm Volume

The Z-800WF Infusion Pump is able to generate multiple audio alert tones to indicate a pump alarm, warning, error or prompt.

Audio alarm volume may be adjusted by the user to Low, Mid or High setting.

- From the configuration menu, Select **2. FACTORY SET**
- In the **FACTORY SET** menu, highlight the **ALARM VOLUME** option by pressing the **Up/Down Arrow**.
- Press the **↑** or **↓** key to adjust alarm volume to **LOW, MID** or **HIGH**.
- Press the **SAVE** **⏏** key to confirm the set value.  
Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the **CONFIGURATION** menu.

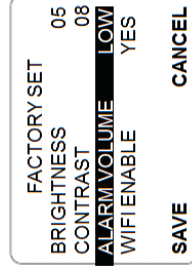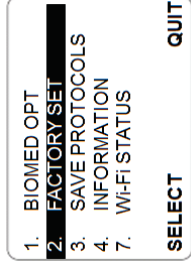


## TIME/VOLUME Programming (Continued)

### Programming in TIME/VTBI Option

Upon selection of the **TIME/VTBI** programming option, the continuous mode infusion programming screen will present **Total Infusion Time** and **Total VTBI** parameters. The **TIME/VTBI** infusion programming screen contains the following display areas:

- Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - CONT:** Continuous Mode Operation
  - VINF:** Volume Infused for the current infusion
- Infusion Parameters:** Primary Infusion Time and Primary Volume to Be Infused.
  - Pri TIME:** Primary Infusion Time
  - Pri VTBI:** Primary VTBI
- Keys:** Allows user to access pump configuration and secondary infusion programming screen.
  - CONFIG:** Accesses pump configuration.
  - SEC:** Accesses secondary infusion programming screen. The Secondary Infusion Parameters will be presented as **Sec TIME** and **Sec VTBI**.
  - Up/Down Arrows:** Select parameter items. Similar to the **RATE/VTBI** programming screen, the **TIME/VTBI** infusion parameters may be programmed by using the **UP/DOWN** arrow keys and **Data Entry** keys.

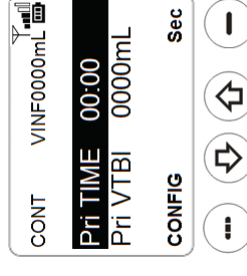

**⚠ WARNING: Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.**

## INFUSION COMPLETE

When the VTBI parameter of the current infusion decreases to zero, the pump determines that the infusion is completed and the pump automatically switches to KVO mode.

The Main display will present the “INFUSION COMPLETED – KVO” message and the KVO flow rate. An audio warning tone will sound every 5 seconds until the user presses the RUN/STOP key. If optional secondary alarm light is connected to the pump, this alarm light will turn solid (blue or red) during KVO mode until user presses the RUN/STOP key.

The pump will continue infusing fluid into the patient with a “Keep Vein Open” rate of 5ml/h. The KVO rate of 5ml/h is also displayed in the Flow Rate LED.

Press the **RUN/STOP** key to acknowledge the infusion complete message. The pump will pause and the KVO infusion will be stopped.

**⚠ WARNING: Make sure to close roller clamp before removing administration set from Z-800WF Infusion Pump.**





## CONFIGURABLE SETTINGS (Continued)

### Adjusting the Contrast

The Z-800WF Main LCD Display’s contrast level can be adjusted by the user.

1. From the configuration menu, Select 2. **FACTORY SET**
2. In the **FACTORY SET** menu, highlight the **CONTRAST** option by pressing the **Up/Down Arrow**.
3. Press the **←** or **→** key to adjust Contrast from 00 to 09.
4. Press the **SAVE** key to confirm the set value.

Press the **CANCEL** key to back out. Press the **QUIT** key again to exit the CONFIGURATION menu.

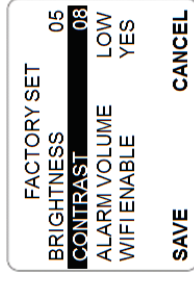
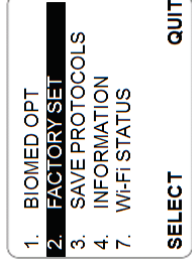


## CONFIGURABLE SETTINGS (Continued)

### Adjusting the LCD Brightness

The Z-800WF Main LCD Display's brightness level can be adjusted by the user.

- From the configuration menu, Select **2. FACTORY SET**
- In the **FACTORY SET** menu, highlight the **BRIGHTNESS** option by pressing the **Up/Down Arrow**.
- Press the **Up** or **Down** key to adjust **BRIGHTNESS** from 00 to 09.
- Press the **SAVE** key to confirm the set value. Press the **CANCEL** key to back out. Press the **QUIT** key again to exit the **CONFIGURATION** menu.

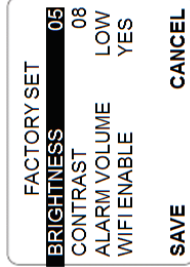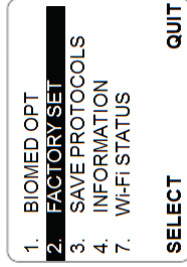

## TPN MODE INFUSION


### Access TPN Mode Infusion

The user may choose the TPN auto ramp infusion mode.

- Press Home key to access the main page. If a confirmation page showing "Current infusion parameters will be lost. Want to continue?" is displayed, select **YES** to acknowledge.

**Note:** Any unsaved parameters will be lost upon confirmation.

- Select **PROGRAM** to access Infusion Mode menu.
- In pump Infusion Mode menu, use the Up/Down Arrow keys to highlight **3. TPN MODE** and press the **SELECT** key.
- The pump will display the Programming screen of the TPN Mode infusion.

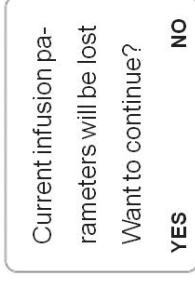


## TPN MODE INFUSION (Continued)

### Programming TPN Mode Infusion

TPN infusion programming screen contains the following display areas:

- Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - TPN:** TPN mode operation
  - VINF:** Volume Infused for the current infusion
- Infusion Parameters:** TPN total VTBI and Total TIME of the infusion.
  - TIME:** Total TIME of the infusion.
  - VTBI:** Total Volume to Be Infused.
- Keys:** Allow user to access pump configuration and TPN ramp programming screen
  - CONFIG:** Accesses pump configuration
  - RAMP:** Accesses TPN Ramp Up and Ramp Down Time parameter programming
  - Up/Down Arrows:** Highlight the selected parameter item

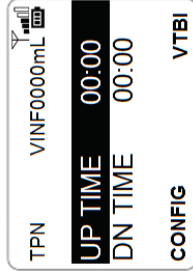

The TPN infusion parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

### Access TPN Ramp Time Parameter Programming

From TPN programming screen, press RAMP Key to access TPN Ramp Parameter programming screen.

### Programming TPN Ramp Time Parameters


The TPN Ramp Time parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

**⚠ WARNING: Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.**


## CONFIGURABLE SETTINGS (Continued)

### Configuring Secondary Infusion Option

The Z-800 pump has the option to hide the Secondary Infusion option so that it is not selectable by the user. To hide or unhide this option, select the **BIOMED OPT Menu**. When hiding the secondary infusion option, it will no longer be selectable by the user.

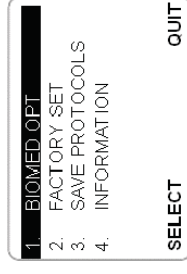

- From the configuration menu, Select (1), **BIOMED OPT**.
- In the **BIOMED OPT** menu, highlight the **HIDE SEC** option by pressing the **Up/Down Arrow**.
  - Press **↶** or **↷** key to switch between **YES** and **NO**.
  - Press the **SAVE** key **⏏** to confirm the set value.
  - Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the CONFIGURATION menu.

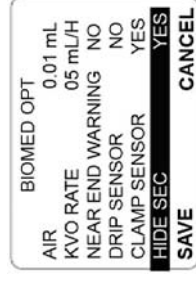

**NOTE:** Hiding the secondary mode should be completed prior to starting an infusion for proper administration. HIDE SEC cannot be changed when ZynoFlo patient query mode. When in manual continuous combined infusion, changing HIDE SEC to YES results in the infusion parameters being zeroed out. When in manual non-continuous infusion, changing HIDE SEC has no effect on selected infusion.

# MAINTENANCE

## CONFIGURABLE SETTINGS (Continued)




### Enable/Disable Clamp Sensor Option

The user may enable or disable the clamp sensor in the configuration menu. When enabled, the clamp sensor will detect the absence of the IV set based free flow protection clamp, alarm and stop the pump when the user tries to start an infusion.

- From the configuration menu, Select **1. BIOMED OPT**.
- In the **BIOMED OPT** menu, highlight the **CLAMP SENSOR** option by pressing the **Up/Down Arrow**.
- Press **↶** or **↷** key will bring out the customization authorization screen.
- Enter the password "1111" by pressing **↶** and **10** once and pressing **100** 11 times. Then press "SELECT." The pump will return to the **BIOMED OPT** menu.
- Use **↶** or **↷** to switch between YES (Enable) and No (Disable).
- Press the **SAVE** key **⏏** to confirm the set value.

Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the **CONFIGURATION** menu.

**⚠ WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.

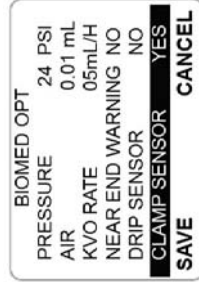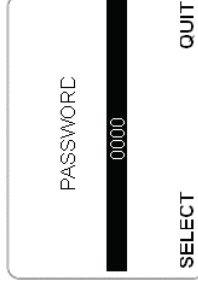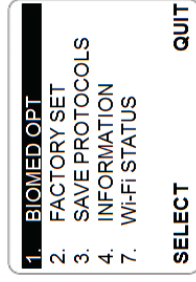

# INFUSION MODE INFORMATION

## 10-STEP MODE INFUSION

### Access 10-STEP Mode Rate/Volume Infusion or 10-STEP Mode Time/Volume Mode Infusion

User may choose either 10 STEP infusion mode.


- Press Home key to access the main page. If a confirmation page showing "Current infusion parameters will be lost. Want to continue?" is displayed, select YES to acknowledge.

Note: Any unsaved parameters will be lost upon confirmation.

- Select **PROGRAM** to access Infusion Mode menu.

- In pump Infusion Mode menu, use the Up/Down Arrow keys to highlight **10 STEP MODE R/V** or **10 STEP MODE T/V** and press the **SELECT** key.

- The pump will display the Programming screen of the 10 STEP Mode infusion.

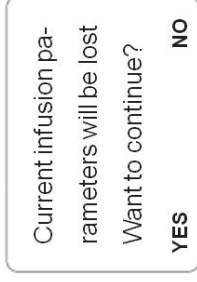


## 10-STEP MODE RATE/VOLUME INFUSION (Continued)

### Programming 10-STEP Mode Rate/Volume Infusion

Similar to the Continuous Mode infusion programming screen, the 10-STEP R/V infusion programming screen contains the following display areas:

1. **Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - **STEP01:** 10-STEP mode operation-Step01
  - **VINF:** Volume Infused for the current infusion
2. **Infusion Parameters:** Step01 Flow Rate and Step01 Volume to Be Infused
  - **RATE:** Current Step Flow Rate
  - **VTBI:** Current Step VTBI
3. **Keys:** Allow the user to access pump configuration and option setup screen.
  - **CONFIG:** Accesses pump configuration
  - **Up/Down Arrows:** Select parameter items and navigate between current step and previous/next step parameter programming

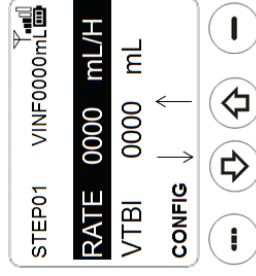

**WARNING:** Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.

### Navigate to Previous/Next Step

Use Up/Down Arrow keys to select current step parameters as well as change to previous or next step programming screen.



Similar to the Continuous Mode infusion programming screen, the 10-STEP infusion parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys. Simply continue to scroll down to program each step. The ten steps are a scrollable list that can be edited anytime during programming. Scroll and carefully review the steps you programmed before pressing **RUN/STOP** to begin the infusion. Pressing **RUN/STOP** to begin the infusion locks in the list. To edit the list after an infusion is underway, press **RUN/STOP** to pause, and then press the **PROG** key to edit the list.

## CONFIGURABLE SETTINGS (Continued)

### Enable/Disable Drip Sensor Option

The Z-800WF pump has an optional external drip sensor, which may be used to detect a fluid container empty event. The user may enable or disable the drip sensor in the configuration menu. To enable or disable this option, refer to “Customize the BIOMED OPT Menu” page in this section. When enabled, the drip sensor will detect the absence of drips in the drip chamber, continue to infuse 8mL, then alarm and stop the pump.

1. From the configuration menu, Select (1), **BIOMED OPT**.
2. In the **BIOMED OPT** menu, highlight the **DRIP SENSOR** option by pressing the **Up/Down Arrow**.
3. Press **←** or **→** key to switch between **YES** and **NO**.
4. Press the **SAVE** key **⏎** to confirm the set value. Press the **CANCEL** key **⏎** to back out. Press the **QUIT** key **⏎** again to exit the **CONFIGURATION** menu.

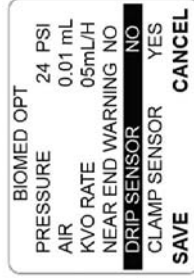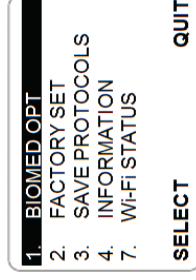



## MAINTENANCE


### CONFIGURABLE SETTINGS (Continued)

#### Enable/Disable NEAR END WARNING

The Z-800WF Infusion Pump has a feature to detect that the current infusion is about to end in 3 minutes. The user may enable or disable the near end warning in the configuration menu. When enabled, the audio and visual warning message will appear every 5 seconds until infusion completion.

- From the configuration menu, Select **1. BIOMED OPT.**
- In the **BIOMED OPT** menu, highlight the **NEAR END WARNING** option by pressing the **Up/Down Arrow**.
- Press **↶** or **↷** key to switch between **YES** and **NO**.
- Press the **SAVE** key **⏏** to confirm the set value. Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the **CONFIGURATION** menu.

**⚠ WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.

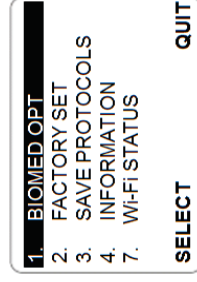

## INFUSION MODE INFORMATION


### 10-STEP TIME/VOLUME MODE INFUSION (Continued)

#### Programming 10-STEP Time/Volume Mode Infusion

Similar to the Continuous Mode infusion programming screen, the 10-STEP TVV infusion programming screen contains the following display areas:

- Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - STEP01:** 10-STEP mode operation-Step01
  - VINF:** Volume Infused for the current infusion
- Infusion Parameters:** Step01 Time and Step01 Volume to Be Infused
  - TIME:** Current Step TIME
  - VTBI:** Current Step VTBI
- Keys:** Allow the user to access pump configuration and option setup screen.
  - CONFIG:** Accesses pump configuration
  - Up/Down Arrows:** Select parameter items and navigate between current step and previous/next step parameter programming

**⚠ WARNING:** Do not over-program VTBI. Program the actual amount of the fluid in the IV bag.

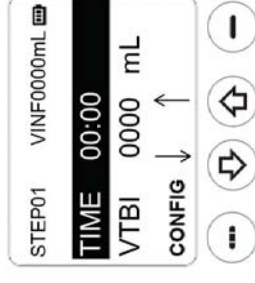

#### Navigate to Previous/Next Step

Use Up/Down Arrow keys to select current step parameters as well as change to previous or next step programming screen.

Similar to the Continuous Mode infusion programming screen, the 10-STEP TVV infusion parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys. Simply continue to scroll down to program each step. The ten steps are a scrollable list that can be edited anytime during programming. Scroll and carefully review the steps you programmed before pressing **RUN/STOP** to begin the infusion. Pressing **RUN/STOP** to begin the infusion locks in the list. To edit the list after an infusion is underway, press **RUN/STOP** to pause, and then press the **PROG** key to edit the list.

## INTERMITTENT MODE INFUSION

### Access Intermittent Mode Infusion

The user may choose the Intermittent Mode from Infusion Mode screen after entering a new infusion

1. Press Home key to access the main page. If a confirmation page showing “Current infusion parameters will be lost. Want to continue?” is displayed, select YES to acknowledge.


**Note:** Any unsaved parameters will be lost upon confirmation.

2. Select **PROGRAM** to access Infusion Mode menu.

3. In pump Infusion Mode menu, use the Up/Down Arrow keys to highlight **5. INTERMITTENT** and press the **SELECT** key.

4. The pump will display the Programming screen of the intermittent mode infusion.

5. In intermittent mode, the pump runs for the period programmed, and then idles before the next period run. The sum of the “run” and the “idle” time is the cycle time, as illustrated below.

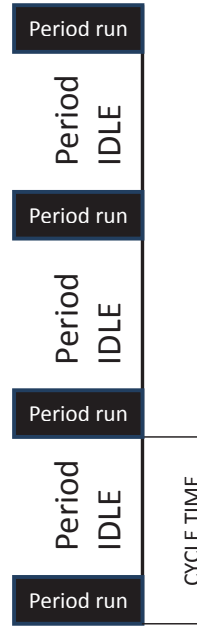

## CONFIGURABLE SETTINGS (Continued)


### Adjusting the KVO RATE

The KVO rate can be adjusted in the configuration menu. The KVO rate can be adjusted by the user from 01ml/h to 20ml/h.

1. From the configuration menu, Select **1. BIOMED OPT.**



2. In the **BIOMED OPT** menu, highlight the **KVO RATE** option by pressing the **Up/Down Arrow**.

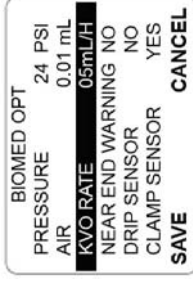

3. Press **←** or **→** key to adjust the value.
4. Press the **SAVE** key **↵** to confirm the set value. Press the **CANCEL** key **⏪** to back out. Press the **QUIT** key **⏩** again to exit the **CONFIGURATION** menu.

**⚠ WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.


## CONFIGURABLE SETTINGS (Continued)

### Adjusting the Air-In-Line Alarm Threshold

The Z-800WF's Air-In-Line sensor detects an air bubble in the administration set tubing. Air-In-Line alarm threshold can be adjusted in the configuration menu. Alarm threshold can be adjusted to air bubble size from a volume of 2 micro-liters to 280 micro-liters. The Z-800WF's Air-In-Line Detector alarm threshold can be adjusted by the user.

1. From the configuration menu, Select **1. BIOMED OPT.**
2. In the **BIOMED OPT** menu, highlight the **AIR** option by pressing the **Up/Down Arrow**.
3. Press **↶** or **↷** key to adjust the value.
4. Press the **SAVE** key **⏏** to confirm the set value. Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the configuration menu.

**⚠ WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.

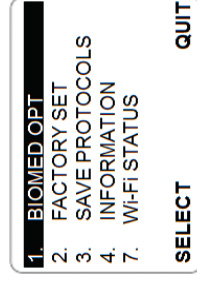

## INTERMITTENT MODE INFUSION (Continued)

### Programming INTERMITTENT Mode Infusion

INTERMITTENT infusion programming screen contains the following display areas:

1. **Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - **INTR:** INTERMITTENT mode operation
  - **VINF:** Volume Infused for the current infusion
2. **Infusion Parameters:** INTERMITTENT total VTBI and Cycle TIME of the infusion.
  - **Total VTBI:** Total Volume to Be Infused
  - **Cycle Time:** This displays the time for each cycle in Hour: Minute format. Cycle time should be programmed between 30 minutes and 24 hours. If the programmed Cycle Time is outside the range, pump alarms “Invalid Parameters.”
3. **Keys:** Allow user to access pump configuration and INTERMITTENT ramp programming screen.
  - **CONFIG:** Accesses pump configuration
  - **PERIOD:** Accesses INTERMITTENT period parameter programming
  - **Up/Down Arrows:** Highlight the selected parameter item



The Total VTBI and Cycle Time parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

**Note:** During idle period, the pump runs at KVO rate. Volume infused during idle period is counted. The pump alarms “Infusion Completed” once VINF (volume infused) equals programmed Total VTBI.

## INTERMITTENT MODE INFUSION (Continued)

### Programming INTERMITTENT Mode PERIOD parameters

INTERMITTENT infusion PERIOD programming screen contains the following display areas:

- Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - **INTR:** INTERMITTENT mode operation
  - **VINF:** Volume Infused for the current infusion
- Infusion Parameters:** INTERMITTENT Period VTBI and Period TIME of the infusion
  - **Period VTBI:** Period Volume to Be Infused. The programmed value of this parameter has to be equal or less than the Total VTBI.
  - **Period Time:** This displays the time for each period run in Hour : Minute format. Period time has to be programmed between 10 minutes and 23 hours. If programmed Period Time is outside the range, pump alarms "Invalid Parameters".
- Keys:** Allow user to access pump configuration and INTERMITTENT TOTAL programming screen.
  - **CONFIG:** Accesses pump configuration
  - **TOTAL:** Accesses INTERMITTENT Total parameter programming
  - **Up/Down Arrows:** Highlight the selected parameter item

The Period VTBI and Period Time parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

**Example:** To infuse total of 200mL drug intermittently, infuse 50mL (Period VTBI) every 6 hours (Cycle Time). The run time for each period is 10 minutes (Period Time), idle time in each cycle is 5 hours 50 minutes. The parameters can be programmed as follows:

Total VTBI: 200mL  
Cycle Time: 06:00


Period VTBI: 50mL  
Period Time: 10 minutes

## CONFIGURABLE SETTINGS

### Adjusting the Occlusion Pressure Threshold

The pressure sensor detects down-stream occlusion in the IV sets. A user may specify the occlusion alarm pressure threshold in configuration screen. The occlusion alarm threshold can be adjusted between 4 psi and 30 psi

- From the configuration menu, Select 1. **BIOMED OPT.**
- In the **BIOMED OPT** menu, highlight the **PRESSURE** option by pressing the **Up/Down Arrow**.
- Press **←** or **→** key to adjust the pressure.
- Press the **SAVE** key **⏏** to confirm the set value. Press the **CANCEL** key **⏏** to back out. Press the **QUIT** key **⏏** again to exit the configuration menu.

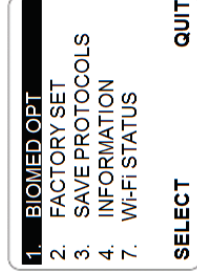

**⚠ WARNING:** The factory default settings should be used unless qualified clinical personnel determine that other customized settings are appropriate and safe.

## MAINTENANCE

### SPECIFICATIONS (Continued)

#### Environmental

#### Non-Operating Conditions (Transportation and Storage):

Temperature: -40°C to 55 °C (-13 °F to + 122 °F)

Humidity: < 93% R.H., non-condensing

Air pressure: 48kPa to 110kPa

#### Operating Conditions

The system may not meet all performance specifications if operated outside of the following conditions:

Temperature: +10°C to +40 °C (+59 °F to + 113 °F)

Humidity: 30 % to 75% R.H. at +40 °C, non-condensing

Air pressure: 70kPa to 106kPa

#### IV Administration Set:

Use only Zyno Medical approved Administration IV Sets.  
Air-in-Line  
Occlusion  
Door Open  
Battery Empty  
No Drip (Optional)  
No Clamp

#### Warnings:

Low Battery  
Pump Unattended  
Near End  
Infusion Complete – KVO  
Due for Service

#### Errors:

System Error

## INFUSION MODE INFORMATION

### BLOOD INFUSION

#### Access **BLOOD INFUSION** mode

The user may choose the BLOOD INFUSION Mode.

1. Press Home key to access the main page. If a confirmation page showing "Current infusion parameters will be lost. Want to continue?" is displayed, select YES to acknowledge.

**Note:** Any unsaved parameters will be lost upon confirmation.

2. Select **PROGRAM** to access Infusion Mode menu.

3. In pump Infusion Mode menu, use the Up/Down Arrow keys to highlight **6. BLOOD INFUSION** and press the SELECT key.

4. The pump will display the Programming screen of the BLOOD INFUSION mode.

Current infusion parameters will be lost  
Want to continue?  
**YES** **NO**

**PROGRAM**  
PROTOCOL  
PATIENT QUERY  
**SELECT**

2. CONT MODE T/V  
3. TPN MODE  
4. 10 STEP MODE R/V  
5. 10 STEP MODE T/V  
6. INTERMITTENT  
**7. BLOOD**  
**SELECT**


## BLOOD INFUSION (Continued)

### Programming Blood Infusion Mode parameters

Blood infusion programming screen contains the following display areas:

- Status Bar:** The Status Bar displays current operating mode, current pump state, and volume infused.
  - **BLD:** BLOOD mode operation.
  - **VINF:** Volume Infused for the current infusion.
- Infusion Parameters:** BLOOD flow rate and Volume to Be Infused.
  - **RATE:** Blood flow rate
  - **VTBI:** Volume to Be Infused
- Keys:** Allow user to access pump configuration programming screen.
  - **CONFIG:** Access pump configuration
  - **Up/Down Arrows:** Highlight the selected parameter item

The RATE and VTBI parameters may be programmed by using the UP/DOWN Arrow keys and Data Entry keys.

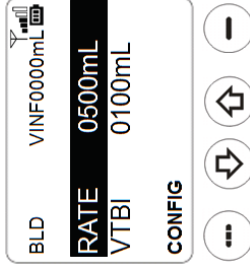

## MAINTENANCE

### SPECIFICATIONS

Pumping Mechanism:	Linear Peristaltic
Flow Rate:	1 - 1200 ml/h in 1 ml increments
Priming Rate:	600 ml/h
Volume to Be Infused:	1 - 9,999 ml in 1 ml increments
Accuracy*:	± 5 %
Time Memory is maintained:	Permanent for previous infusion parameters.
Air-in-Line Detector:	Ultrasonic, 14-level configurable
Occlusion Detector:	Pressure sensor. 14-level configurable
Electrical Standard:	Class I, Type BF
Electrical Safety:	Complies with: EN 60601-2-24:1998; EN 60601-1:2006/AC:2010; EN 60601-1-2:2007/AC:2010; EN 60601-1-6:2010; EN 60601-1-8:2007; EN 60601-1-11:2010, and EN 62366:2008.
Power Requirements:	100 – 240 VAC, 50-60 Hz
Power Consumption:	25 VA (at maximum flow rate)
Internal Battery:	Rechargeable Nickel-Metal-Hydride 9.6V; 4.5 Amp-hr (Replaceable by qualified service personnel only)
Battery Life at 125 ml/h:	8 hours
Battery Charging:	Automatic when pump plugged into an AC power source
Pump Housing:	Cast Aluminum & Sheet Metal
Weight:	3.2 kilograms, 7.0 lbs
Dimensions:	8.6”H x 5.7”W x 5.3”D
Standards:	Manufactured in a facility certified with ISO 13485: 2003

\*The device is calibrated within flow rate accuracy specification under the following conditions:

- Test fluid is distilled water at room temperature (21±/1°C) at ambient air pressure;
- Head height is set according to instructions in the IFU;
- Test set is Zyno proprietary set at specifications;
- Programmed volume during test is no less than 10 mL.




## ERRORS

Error	Meaning	Secondary Alarm/ Warning Light (optional)	Response
System Error	The system has detected an error in the pump. Infusion stops.	Continuously flashes (blue or red) notification until user acknowledgement.	Press <b>RUN/STOP</b> key to silence the alarm and to stop using the affected pump.  The pump must be taken out of service until serviced by qualified service personnel.

## MANAGING INFUSION

### PRIME THE ADMINISTRATION SET USING PRIME KEY

The user can use the **PRIME** key to prime air bubbles out of the IV Administration Set from the distal end after it is loaded in the pump.

1. Press the **PRIME** key. The pump screen will display the PRIME warning message “Disconnect patient, then press and hold soft “Prime” key as indicated below.” Confirm that the patient is not connected to the administration set. Press and hold the  key to prime.
2. During the prime, the pump will run at the fixed prime flow rate of 600mL/hr. Both Air-In-Line sensor and Occlusion sensor are disabled.
3. Release the  key when the set is fully primed. While the  key is pressed, the pump screen will report the volume primed. The maximum prime volume is 10mL. Upon reaching the maximum prime volume, the prime will stop.



**⚠ WARNING: Patient must be disconnected before utilizing the PRIME key.**

## KEYPAD LOCKOUT

To lock keypad, press the CLR key and immediately after that the 10's DOWN data entry key. An audio prompt tone will sound and a "PANEL LOCKED" message will be displayed for 3 seconds.

1. During KEYPAD LOCKOUT, the user may only press and hold ON/OFF key for emergency shut off.
2. Any other key press will result a message display of "PANEL LOCKED" for 3 seconds.
3. To unlock the keypad, press the CLR key and immediately after that the 10's DOWN data entry key. An audio prompt tone will sound and a "PANEL UNLOCKED" message will
4. be displayed for 3 seconds.



## WARNINGS (Continued)

<p>Infusion Complete – KVO</p>	<p>Infusion is completed. The pump automatically switches to KVO mode.  The visual warning message will appear and the audio alarm will sound constantly until user acknowledgement.</p>	<p>Stays solid (blue or red) notification until user acknowledgement.</p>	<p>Press the RUN/STOP key.  The Z-800WF Pump will be paused.</p>
<p>Pump will be due for service soon.  Pump is due for service.</p>	<p>The pump needs to be serviced soon.  The pump must be serviced before use.</p>	<p>N/A  N/A</p>	<p>Call 1-866-395-1988 to schedule the pump maintenance service.  Stop using the pump. Call 1-866-395-1988 to schedule the pump maintenance service.</p>



## WARNINGS

Warning	Meaning	Secondary Alarm/ Warning Light (optional)	Response
Low Battery	Z-800WF Infusion Pump is operating on battery power and low battery condition is detected. Battery has 30 minutes or less of power at current rates before operation will stop.  The infusion continues.  The audio and visual warning message will appear every 5 seconds.	Continuously flashes (blue or red) notification until user acknowledgement.	Plug the power cord into an AC power outlet. The audio warning will be silenced when AC power is detected.
Pump Unattended	The Z-800WF Infusion Pump detects the current infusion is paused and there is no user input for more than 5 minutes.  The audio and visual warning message will appear every 5 seconds until user acknowledgement.	Continuously flashes (blue or red) notification until user acknowledgement.	Press the <b>RUN/STOP key</b> .  The Z-800WF Pump will display the current infusion programming screen.
Near End	The Z-800WF Infusion Pump detects the current infusion is about to end in 3 minutes.  The audio and visual warning message will appear every 5 seconds until infusion completion.	Continuously flashes (blue or red) notification until user acknowledgement.	Prepare for completion of the infusion which will occur in 3 minutes.

## CLEAR VOLUME INFUSED

- From primary or secondary setup state, user may elect to clear the current volume infused (VINI) parameter.
- Press the RUN/STOP key to PAUSE the primary or secondary infusion during infusion.
- Press the PROG key to return to programming page, press and hold **CLR** key, an audio prompt tone will sound and a confirmation screen will be displayed.
  - Press the **YES** key to confirm the action
  - Press the **NO** key to abort the action.
- Upon user confirmation, the current VINI parameter will be reset to 0 ml.



**NOTE:** The VINI field will record cumulative infusion volume infused unless cleared by pressing the **CLR** key.

## MANAGING INFUSION

### POWERING OFF

Stop the Z-800WF Infusion Pump by pressing the **RUN/STOP** key during an infusion.

Press and hold down the **ON/OFF** key for 2 seconds. The pump will turn off.



## ALARMS AND TROUBLESHOOTING

### ALARMS (Continued)

No Clamp	The clamp sensor does not detect the presence of the IV set based clamp in pump. The infusion is stopped.	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>1. Acknowledge the alarm by pressing <b>RUN/STOP</b> key.</li> <li>2. The infusion is paused.</li> <li>3. Load the IV set based clamp correctly. Press the <b>RUN/STOP</b> key again to resume infusion.</li> </ol>
----------	--	---	--

## ALARMS

Alarm	Meaning	Secondary Alarm/ Warning Light (optional)	Response
Air-in-line	Air has been detected in the set during an infusion. Infusion stopped.	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>Acknowledge the alarm by pressing <b>RUN/STOP key</b>.</li> <li>The infusion is paused.</li> <li>Clear air from line and press the <b>RUN/STOP key</b> again to resume infusion.</li> </ol>
Door Open	Pump door is opened during an infusion. The infusion is stopped.	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>Acknowledge the alarm by pressing the <b>RUN/STOP key</b>.</li> <li>The infusion is paused.</li> <li>Close the pump door. Press the <b>RUN/STOP key</b> again to resume infusion.</li> </ol>
Occlusion	Increased back pressure is sensed while infusing. Infusion is stopped.	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>Resolve the cause of the occlusion.</li> <li>The infusion will resume when pressure is reduced to below the alarm threshold.</li> </ol>
No Drip	The drip sensor does not detect a fluid drop in the drip chamber (continues for 8mL, then alarms).	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>Acknowledge the alarm by pressing <b>RUN/STOP key</b>.</li> <li>The infusion is paused.</li> <li>Verify that the fluid container is not empty. Press the <b>RUN/STOP key</b> again to resume infusion.</li> </ol>
Battery Empty	The Z-800WF pump is operating on battery power and battery is too low for pump operation.	Continuously flashes (blue or red) notification until user acknowledgement.	<ol style="list-style-type: none"> <li>Plug the power cord into an AC power outlet.</li> <li>Acknowledge the alarm by pressing <b>RUN/STOP key</b>.</li> <li>The infusion is paused. Press <b>RUN/STOP key</b> again to resume infusion.</li> </ol>


## CHANGING PRIMARY SOLUTION CONTAINER

- Stop the Z-800WF Infusion Pump by pressing the **RUN/STOP** key during an infusion.
- Close the Roller Clamp.
- Remove the empty solution container.
- Spike the new container.
- Go to the primary infusion programming screen. Highlight Pri VTBI parameter. Use the data entry keys to enter desired VTBI.
- Open the roller clamp.
- Press the **RUN/STOP** key to resume infusion.


## MANAGING INFUSION

### CHANGING AND RELOADING IV SET DURING INFUSION

1. Stop the Z-800WF Infusion Pump by pressing the **RUN/STOP** key during an infusion.
2. Close the Roller Clamp.
3. Disconnect the IV Administration Set from the patient.
4. Open the Pump Door.
5. Release the IV Administration Set from the Free-Flow Clamp located at the bottom portion of the pump.
6. Push the metal button of the Anti-Free-Flow Module, slide the Anti-Free-Flow Clamp out and release the button.
7. Prime and load a new IV Administration Set, as described in the Prepare Infusion section.
8. Close the Pump Door.
9. Open the Roller Clamp.
10. Press the RUN/STOP key to resume infusion.

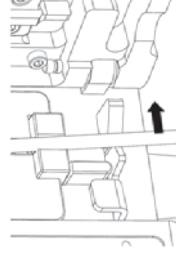

Step 5

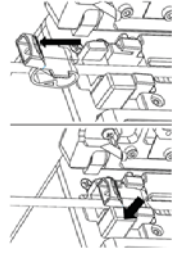

Step 6

## ALARMS AND TROUBLESHOOTING

### ALARMS AND TROUBLESHOOTING

To enhance safety and ease of operation, the Z-800WF Infusion Pump provides a full range of audio and visual alarms, warnings, and prompts.

#### DEFINITIONS

**Error** An audio and visual signal indicates a failure has been detected. Immediate action is required.

The affected Z-800WF pump needs to be replaced with an operational unit. The affected pump should be serviced by qualified personnel.

**Alarm** An audio and visual signal indicates that a potentially unsafe condition is detected. Immediate action is required.

Under an alarm condition, the Z-800WF Infusion Pump is in STOP state. The audio signal will sound until positive confirmation from user is delivered.

The Z-800WF Infusion Pump will not allow the user to resume the infusion until the potentially unsafe condition is resolved.

**Warning** An audio and visual signal indicates that a potentially unsafe condition is present. Immediate action is required.

Under a warning condition, the Z-800WF Infusion Pump will continue to operate. The audio signal will not be silenced until the warning condition is resolved.




## USING 10-STEP MODE WITH ZYNOFLO

A standardized protocol can be created for drugs with 10-Step Mode infusion using ZynoFlo and the Drug Protocol Tool. This tool allows the user to set a 10-Step Mode Rate/Volume infusion. If the user requires a 10-Step Mode Time/Volume infusion to be programmed, the user can access Time/Volume with the following steps.

**NOTE:** Once 10-Step Mode Time/Volume has been selected the user will not be able to move back and forth between 10-Step Mode R/V and 10-Step Mode T/V.

### Changing 10-Step Rate/Volume Mode to Time/Volume Mode

Prior to running a Patient Query and selecting a drug to be infused the user should configure the mode menu options for 10-Step Mode Time/Volume. Once the 10-Step Mode Rate/Volume has been hidden the pump will only run 10-Step Mode Time/Volume for ZynoFlo created standard protocols. The 10-Step Mode R/V must be hidden or else all protocols will run in 10-Step Mode R/V as the default.

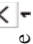1. Select **NEW INFUSION**.
2. Select **PROGRAM**.
3. From the infusion mode screen press and hold CLR key and  key at the same time.
4. Enter password 1111.
5. Highlight 10 Step Mode R/V using  arrow.
6. Use the  key until the asterisk is next to the mode.
7. Press **SELECT** to confirm.



## PROTOCOL MODE INFUSION

### Access Protocol Mode Infusion



A user may choose to save the current infusion parameters to the pump as a pre-defined protocol. The saved protocol can be retrieved in the future to automatically program a current infusion. This can eliminate repetitive programming for selected commonly used infusion regimens.

The Protocol infusion mode may be accessed after entering a new infusion.

1. Turn on the pump and select **NEW INFUSION**, or, if the pump is in infusion parameter programming screen, press Home key to access home page.
2. The home page should be displayed. Highlight **PROTOCOL** using Up/Down arrows and press **SELECT**.
3. The pump will display the protocol names saved on the pump. There are up to 8 protocols that can be saved on the pump.
4. Choose one of the protocols by highlighting the name of the protocol. Press the **SELECT** key to program the highlighted protocol.
5. The pump will display the Programming screen of the pre-defined infusion mode with pre-defined infusion parameters populated.

### Start Infusion

Verify the displayed infusion parameter entries (Primary Rate, Primary VTBI, Secondary Rate, and Secondary VTBI). If a clamp is engaged, remove the clamp. Press **RUN/STOP** key to start the infusion.

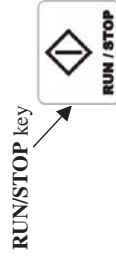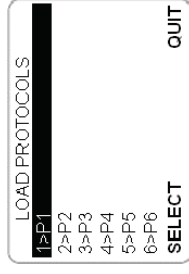

## MANAGING INFUSION

### SAVE INFUSION PARAMETERS AS PROTOCOL




Turn on the pump; Select NEW INFUSION; Select PROGRAM and choose infusion mode to program an infusion.

Example:

1. Select **10 STEP MODE**. Program a 10 Step infusion with the following parameters.
  - **STEP 1:** RATE=50ml/h, VTBI=25ml.
  - **STEP 2:** RATE=100ml/h, VTBI=50ml.
  - **STEP 3:** RATE=150ml/h, VTBI=75ml.
  - **STEP 4:** RATE=200ml/h, VTBI=100ml.
  - **STEP 5:** RATE=250ml/h, VTBI=250ml.


2. From the 10 STEP Programming screen, press the **CONFIG** key to access CONFIG screen. Select **3. SAVE PROTOCOLS**.

3. Select one of the 8 protocols to be used to store the infusion parameters. Enter the name to be used for the protocol.

- Use the **100** or  key to scroll through letter A through Z.
- Use the **10** or  key to scroll through letter A through Z.
- Use the **1** or  key to scroll through digit 0 through 9, SPACE and other special characters.
- Use the Up/Down Arrow to move the cursor back and forth.

4. Press **SAVE** key to associate the current programmed infusion parameters with the protocol name as entered.

**NOTE:** Loading factory default settings will delete all saved protocols. (Refer to Maintenance->Configurable settings->Load default on how to load factory default).

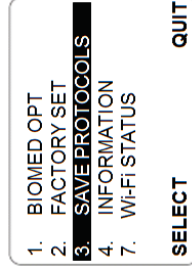

## ZYNOFLO INFORMATION

### MANAGING REVIEWER ID (CAREGIVER ID) REQUIREMENT FOR INFUSIONS

#### Enable Reviewer ID Tracking

As part of the ZynoFlo implementation process a Super User or Zyno representative will add a new user with permissions/access, username, password and user number (Caregiver ID). The Caregiver ID is used to track the clinician giving a specific infusion, a clinician verifying the infusion or a clinician that is changing the parameters of an ongoing infusion. Each user will have a unique Caregiver ID associated with their user account in ZynoFlo.

The requirements for the number of Reviewer IDs requested per infusion can be set on the infusion pump.



#### Reviewer ID Settings

The pump can be set to one of the following options by going to 1. BIOMED OPT and select REVIEWER ID. After entering REVIEWER ID the user can select **Once or Always** by using the  or  key.

The Caregiver ID has two setting options for user interaction;

**Once:** The Reviewer ID will only be requested once at the start of the infusion. The Reviewer ID must be set initially for the specific drug in the ZynoFlo application.

**Always:** The Reviewer ID is requested at the start of the infusion and at any time a change is made to that infusion until completed. The Reviewer ID must be set initially for the specific drug in the ZynoFlo application.

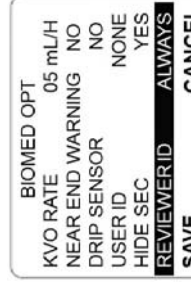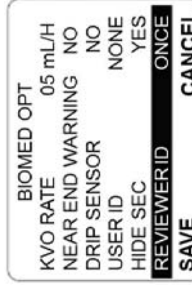

## ZYNOFLO INFORMATION

### ENABLING CAREGIVER ID FOR INFUSIONS (Continued)

#### Encounter - Entering Caregiver ID with Patient Query

A minimum of the Caregiver ID will be required at the start of every patient encounter.

- 1 Start a New Infusion.
- 2 Complete a Patient Query.
- 3 Enter Caregiver ID (Caregiver ID is required).
- 4 If applicable enter Reviewer ID.
- 5 Set and verify modes and parameters or verify pre-populated parameters.
- 6 Start Infusion.

#### Infusion - Entering Caregiver ID with Patient Query

A minimum of the Caregiver ID will be required at the start of every drug infusion.

- 1 Start a New Infusion.
- 2 Complete a Patient Query.
- 3 Select the appropriate drug from the dispense order displayed on the pump.
- 4 Set and verify modes and parameters or verify pre-populated parameters.
- 5 Enter Caregiver ID (Caregiver ID is required).
- 6 If applicable enter Reviewer ID.
- 7 Start infusion.

**NOTE:** If the infusion parameters are changed during an infusion the pump will request a Caregiver ID and Reviewer ID if set from the new users making the change.

## ZYNOFLO INFORMATION

### ZYNOFLO WIRELESS COMMUNICATION CONNECTION

#### Wireless Connection

User may choose to use wireless function of Z-800WF pump under wireless connection. Zyno provided wireless access point should be installed in pump working environment and connected to Ethernet by authorized personnel. For network and system requirement, please refer to Application Programming Interface IFU.

To access the Wi-Fi status information, select 7. Wi-Fi STATUS in CONFIG screen to access the Wi-Fi status. During pump operation, a wireless connection strength sign is shown in the upper right corner of the screen. The antenna sign indicates the pump is connected to the server and the signal bars indicates the signal strength.

#### ZynoFlo System Operation

The Zyno Gateway server application is Java based application server that is intended to provide the electronic transfer, storage, and electronic conversion of data to, from and in association with support Zyno Medical infusion pumps used in oncology centers, hospital or hospital type environments for the purpose of transferring pump information. This Gateway Server communicates with every registered compatible Zyno infusion pump via TCP and/or UDP in the wired/wireless network.

Zyno Gateway server application manages communications between the external drug dispense systems, electronic medical records and the Zyno Medical infusion pumps. The ZynoFlo web based application provides a means for the administrator, pharmacist or nurse to view patient information and manage the medication administration workflow.

## ZYNOFLO INFORMATION

1. BIOMED OPT	<b>SELECT</b>	<b>QUIT</b>
2. FACTORY SET		
3. SAVE PROTOCOLS		
4. INFORMATION		
7. Wi-Fi STATUS		

IP	XXX.XXX.XXX.XXX	QUIT
GW	XXX.XXX.XXX.XXX	
WVF	XXXXXXXXXX	
SSID	XXXX	
MAC	XX:XX:XX:XX:XX:XX	
RSSI	XX%	

CONT	VINF0000mL	
Pri RATE	0125mL/H	
Pri VTBI	0000mL	
<b>CONFIG</b>		<b>Sec</b>

PT confirmation	<b>NO</b>	
Doe, Joe		
MM/DD/YYYY		
<b>YES</b>	<b>NO</b>	
ENTER CAREGIVER ID		
000		
<b>SELECT</b>		<b>QUIT</b>

ENTER REVIEWER ID		
000		
<b>SELECT</b>		<b>QUIT</b>

NS 0.9% 150ML	VINF0000mL	
CONT	VINF0000mL	
Pri RATE 0100 mL/H		
Pri VTBI 0010 mL		
<b>CONFIG</b>		

ENTER CAREGIVER ID		
000		
<b>SELECT</b>		<b>QUIT</b>

ENTER REVIEWER ID		
000		
<b>SELECT</b>		<b>QUIT</b>

## ZYNOFLO INFORMATION

### ZYNOFLO WIRELESS COMMUNICATION CONNECTION (Continued)

#### ZynoFlo System Operation, (continued)

In conjunction with external drug dispense applications the Zyno Gateway Application provides drug preference information for the purpose of verification of the drug orders and patient information on Zyno infusion pumps. The Zyno Gateway Application provides a database for the Zyno pump to query the patient drug orders and display them at the pump. The drug preference functionality may be used by pharmacist and/or nurse to choose drugs to display or to set site specific drug protocols sent to the pump.

The server collects and records all relevant pump state change events and the corresponding pump state information in its database. Information on drug start and stop times can be published to external destinations such as, electronic medical record applications, from the registered Zyno pumps. Within the Zyno Gateway Application it is able to publish relevant events from all connected pumps and create its own billing record for patients infused. In addition the Gateway can enable nurses to query and view the status of specified Zyno infusion pumps further display of patient care.

Note: If the wireless system is chosen, it allows for the exchange of data between the pump and the Zyno provided wireless access point. The pump can be operated manually or in conjunction with the wireless network. If communication with the wireless network experiences extended wireless inactivity the pump can be used as planned without the wireless function.



## ZYNOFLO INFORMATION

### ENABLING CAREGIVER ID FOR INFUSIONS

#### Enable Caregiver ID Tracking

As part of the ZynoFlo implementation process a Super User or Zyno representative will add a new user with permissions/access, username, password and user number (Caregiver ID). The Caregiver ID is used to track the clinician giving a specific infusion, a clinician verifying the infusion or a clinician that is changing the parameters of an ongoing infusion. Each user will have a unique Caregiver ID associated with their user account in ZynoFlo.

#### Caregiver ID Settings

The pump can be set to one of the below options by going to 1. BIOMED OPT and select USER ID. After highlighting USER ID the user can select **None**, **Encounter** or **Infusion** by using the  or  key.

The Caregiver ID has three setting options for user interaction;

**-None:** The Caregiver ID is not active.



**-Encounter:** The Caregiver ID is required after the completion of the patient query process only.

**-Infusion:** The Caregiver ID is required after every drug infusion selected.

NOTE: If Infusion has been enabled, the option to include a Caregiver ID to verify (Reviewer ID) an encounter or infusion must be set by the Drug Preference Tool in the ZynoFlo application. See Managing Reviewer ID (Caregiver ID) Requirement for Infusions.

NOTE: In the PROGRAM mode, only one Caregiver ID will be entered if the pump is enabled for Encounter or Infusion.

NOTE: A Caregiver ID must be entered when requested. The pump will not accept "000" as an ID.

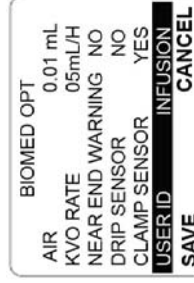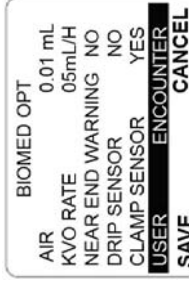



## ZYNOFLO INFORMATION

### OPTIONAL: MANUAL COMMUNICATION WITH ZYNOFLO (Continued)

- The user can refresh the medication dispense order by selecting **Refresh order**. The order list will be refreshed if there is any update.
- When all designated infusion orders are executed, the user may choose **Exit Encounter**. This will bring up the Resume Infusion/New Infusion screen.

In the infusion order list, there are a few notations following drug information:

- (P): Primary Bag in CONT mode
- (S): Secondary Bag in CONT mode
- (M): Mixed/combined drug
- (PM): Primary mixed bag in CONT mode
- (SM): Secondary mixed bag in CONT mode

A \* sign before drug information indicates the order has been executed

Doe, Joe * NS 0.9% 150ML (P) ALOXL1.25MG SOL (S) user-defined 01 Exit Encounter Refresh order	<b>QUIT</b>
<b>SELECT</b>	

	<b>RESUME INFUSION</b>	<b>QUIT</b>
	<b>NEW INFUSION</b>	
	<b>SELECT</b>	
		

## ZYNOFLO INFORMATION

### ZYNOFLO AND PATIENT QUERY COMMUNICATION

User may choose to query medication dispense order list of a specific patient from medication dispense system by using the **PATIENT QUERY** option on Z-800WF infusion pump.

- Turn on the pump and select **NEW INFUSION**, or, if the pump is in infusion parameter programming screen, press Home key to access home page.
- The home page should be displayed. Highlight **PATIENT QUERY** using Up/Down arrows and press **SELECT**.
- The pump will prompt user to enter the last 3 digits of patient ID. Enter by pressing the data entry keys and press **SELECT**. Otherwise, pressing **QUIT** will bring the user back to home page.
- The pump will start searching matching patient ID(s) in the drug dispense system. If no matches are found, the pump will display “NO PATIENT LIST”. Select **QUIT** to go back to home page. If matching patient(s) is found, the pump will provide a matching list with last 4 digits of patient ID(s) and patient name(s). Select correct patient name by highlighting it and pressing **SELECT**.
- The pump will load parameter of selected patient. If no encounter orders are found, the pump will display “NO ENCOUNTER ORDERS”. Select **QUIT** to go back to home page. If the pump displays “NO INTERNET CONNECTION”, check the internet connection. Otherwise the pump will display Patient confirmation page with patient name and DOB. Confirm with patient and information on the drug bag. Select **YES** if the information is correct.

PROGRAM PROTOCOL	<b>SELECT</b>
<b>PATIENT QUERY</b>	

ENTER PATIENT ID LAST 3 DIGITS XXXX 123	<b>QUIT</b>
<b>SELECT</b>	

NO PATIENT LIST	<b>QUIT</b>
-----------------	-------------

PATIENT ID.NAME 8123 DOE, JOE 1123 SMITH, TOM 1123 PARKER, JIM 6123 DAVIS, KATE 9123 CLARK, BRUCE	<b>QUIT</b>
<b>SELECT</b>	

PT confirmation Doe, Joe MM/DD/YYYY	<b>NO</b>
<b>YES</b>	



## ZYNOFLO INFORMATION

### ZYNOFLO AND PATIENT QUERY COMMUNICATION (Continued)

- If the selected order is pre-programmed in the system, the infusion parameters will display with the selected drug information on top line. Confirm the parameters and start the infusion by pressing RUN/STOP button.
- If the selected order is not pre-programmed in the system, the pump will prompt the user to select infusion mode and primary/secondary bag where applicable and input the infusion parameter. Confirm the parameters and start the infusion by pressing RUN/STOP button.
- After the completion of current infusion, press home key at the infusion parameter program screen to get back to the medication dispense order list.
- The user may choose to continue another dispense order by highlighting and selecting appropriate order.
- Or the user may choose to define an infusion order by highlighting and selecting **User-defined XX**. An Infusion Mode screen will display to continue the process.
- The user can refresh the medication dispense order by selecting **Refresh order**. The order list will be refreshed if there is any update.
- When all designated infusion orders are executed, the user may choose **Exit Encounter**. This will bring up the Resume Infusion/New Infusion screen.

In the infusion order list, there are a few notations following drug information:

- (P): Primary Bag in CONT mode
  - (S): Secondary Bag in CONT mode
  - (M): Mixed/combined drug
  - (PM): Primary mixed bag in CONT mode
  - (SM): Secondary mixed bag in CONT mode
- A \* sign before drug information indicates the order has been executed.

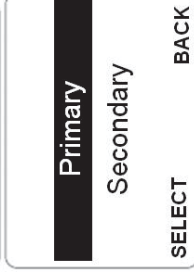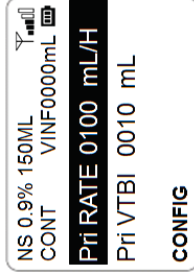

## ZYNOFLO INFORMATION

### OPTIONAL: MANUAL COMMUNICATION WITH ZYNOFLO Send Encounter from ZynoFlo to Pump (optional)

User may log in client account through the ZynoFlo application and send encounter(s) to specific active Z-800WF infusion pump(s) that is properly connected to the wireless access point. Please refer to the ZynoFlo User's Manual for instructions.

- Once an encounter is successfully sent, a confirmation page with patient's name and date of birth will be displayed on pump screen and an audio signal will sound.
- Confirm with patient and label on drug bag with the name and DOB on screen. If the information is right, select **YES**. Otherwise select **NO** to abort the encounter.
- Upon confirmation, the medication dispense order list will display. Review and highlight the right drug and press **SELECT**.
- Pre-programmed infusion parameters will display with the selected drug information on top line. Confirm the parameters and start the infusion by pressing RUN/STOP button.
- After the completion of current infusion, press home key at the infusion parameter program screen to get back to the medication dispense order list.
- The user may choose to continue another dispense order by highlighting and selecting appropriate order.
- Or the user may choose to define an infusion order by highlighting and selecting **User-defined XX**. An Infusion Mode screen will display to continue the process.

