

Cellerate[®]_{RX}

ACTIVATED COLLAGEN[®]

Description

CellerateRX Hydrolyzed Collagen is Type I bovine hydrolyzed collagen and contains no additives. The powder is hydrolyzed collagen and the gel is approximately 65% hydrolyzed collagen and 35% purified water.

Indications

For use in management of chronic and acute wounds including partial- and full-thickness wounds, pressure injuries 1-4, venous stasis ulcers, arterial ulcers, diabetic ulcers, surgical wounds, traumatic wounds, first- and second-degree burns, and superficial wounds.

When to use CellerateRX Gel

Use the gel on dry or lightly to moderately exudative wounds.

Initial Application

It is recommended that the wound bed be debrided of slough and necrotic tissue. Cleanse the wound per facility protocol. The product should cover the entire wound bed and the edges of the wound, including any areas of undermining or tunneling. Cover with an appropriate dressing per facility protocol that provides a moist healing environment. Infected wounds are treated per physician direction.

Reapplication

Reapply CellerateRX minimum 2-3 times per week, or with each dressing change, taking care not to disrupt the wound site.

Contraindications

Known sensitivity to bovine collagen.

Glossary



Single-patient multiple use



Consult instructions for use



Sterile until opened



Do not use if package is damaged



Store at room temperature

Manufactured in the USA for:

Wound Care Innovations, LLC,
a Sanara MedTech Inc. Company
1200 Summit Ave., Suite 414
Fort Worth, TX 76102
800-205-7719

CellerateRX is a
registered trademark of
Applied Nutritionals, LLC
© 2022 Rev. 220623

110228