

## B. Braun Vista Basic Tips for Optimum Usage



INFUSION MADE EASY™

Call 1-800-658-5582 for troubleshooting assistance.



### **Tip #1: Keep the Pump Plugged in at All Times**

To maintain the pump's full battery life, make sure it is plugged in and charging at all times. You can tell if the pump is charging because a small plug-in symbol will show on the pump's screen, similar to this: 

If a patient leaves his/her chair with the pump for any reason, make sure the pump is plugged back in when the patient returns!

If your pump gives you a low battery message, plug it in and let it charge continuously for 24 hours; then check the pump again.

### ***About the Vista Basic's battery:***

Using the pump while it's unplugged should be reserved for emergencies. The internal battery can only operate the pump for 3.5 hours! When the internal battery is depleted, all sorts of random, false errors can occur, including occlusion errors. So please remember to keep the pump plugged in at all times!

### **Tip #2: Protect the Pump from Accidental Impact**

Accidental impact can be caused by dropping the pump, the pump bumping a chair or wall corner, or the pump falling off its pole. These kinds of impact can cause false or inaccurate occlusion alarms; battery damage and/or disconnection from board; main board damage; external damage to handle, door lever, screen, case and feet.

Firmly attach the pump to the pole, about waist height, and make sure the pole has proper stability controls. The higher the pump is on the pole, the more top heavy and likely to tip over it is. Treat the pump with care!

### **Tip #3: Ensure the Door is Closed Correctly**

The pump's door hinge is a weak point on the pump. If the door is closed correctly, damage from accidental impact can be greatly reduced.

### **Tip #4: Use the Proper Set**

Using the correct B. Braun set prevents nuisance alarms. If you wonder if you are using an appropriate set for your infusion requirements, call B. Braun at 800-627-7867.

**Please call an InfuSystem Biomedical Technician if you have a question about your pump, at any time. We can't offer you clinical advice, but we are experts in troubleshooting!  
You can reach us at 800-658-5582.**